

Product Technical Data Sheet 017 Mowbray Lite Toilet Frame & Seat

Product Range

M66625	Mowbray Lite Toilet Frame & Seat — Standard
M92740	Mowbray Lite Toilet Frame & Seat — 510
M66613	Mowbray Lite Toilet Frame & Seat Width Adjustable — Flat Pack
N36974	Mowbray Lite Toilet Frame & Seat Width Adjustable — Assembled

Product Description & Intended Purpose

To assist those who have difficulty lowering or rising from a standard toilet.

Product Features

- Smooth and contoured plastic seat provides extra comfort and thigh support.
- Shaped seat to aid the rising and lowering motion.
- Non slip rubber ferrules to provide stability.
- Chute incorporated in seat design to simplify disposal.



Technical Specification

Dimensions (mm)

- | | |
|--|---|
| <ul style="list-style-type: none"> • Seat Height—400—550 • Seat Width X Depth—406 X 450 • Seat Aperture— 260 X 220 • Frame Height—610—770 • M66625—Width between arms—450
Footprint—520 X 515—565 | <ul style="list-style-type: none"> • M92740—Width Between Arms—510
Footprint—627 X 515—565 • M66613—Width Between Arms—510—660
Footprint—570—720 X 515—565 • N36974—Width Between Arms—510—660
Footprint— 570—720 X 515—565 • Maximum User Weight—190Kg (30st) |
|--|---|

Installation Information *(Note: for detailed information refer to User Instructions)*

- Push toilet seat onto frame by clipping over tube.
- To adjust height and width, remove 'e-clips', slide inner tube to desired position, align holes and re-insert 'e-clips'.

PTDS017

Mowbray Lite Toilet Frame & Seat

Safety Precautions

- Avoid extending the legs in a manner which causes the seat to be on a slant.
- Do not move the frame whilst the seat is occupied.
- Do not exceed the maximum user weight of 30st 190kg
- Avoid using the product on a wet or slippery floor.
- Do not attempt to stand on the frame or seat.

Cleaning

- Clean regularly, using a general purpose, neutral detergent and warm water.
- Please check the suitability of the cleaning product before use. Do not use abrasive cleaners or cloths as this will permanently damage the surface and invalidate any warranty.
- To disinfect, wipe with a dilute bleach solution, and rinse thoroughly with fresh water.
- The product is suitable for thermal decontamination and can be steam cleaned or autoclaved on cleaning cycles up to 70°C.
- Ensure the seat is dried thoroughly before stacking & storage.

Care & Maintenance

- Inspect the product regularly for signs of wear or damage, pay particular attention to the condition of the feet/ferrules.
- Check that all fixing clips are inserted correctly and there is no excessive movement.
- Check for general wear or damage. Any damage should immediately be reported to the supplier and the product taken out of action.

Accessories / Related Products

F11572—Floor Fixing Kit for Mowbray Toilet Frames

F17938—Spare Ferrule

FAQ's:

Product Supplied By:

NRS Healthcare Sherwood House, Cartwright Way, Forest Business Park, Bardon Hill, Coalville, Leicestershire. LE67 1UB UK