




	NEW CODE	CODE
Raised Toilet Seat	09 107 9185	AA2112
	09 107 9276	AA2114Y
	09 107 9300	AA2116
Raised Toilet Seat With Lid	09 107 9193	AA2112L
	09 107 9243	AA2114L
	09 107 9318	AA2116L

Intended uses

- Toilet aids allow for, or increase, independence when using the toilet, particularly by compensating for impaired movement

Instructions for installation

1. Raise the toilet seat that is attached to the toilet. The Savanah® Raised Toilet Seat should be put onto the toilet bowl.
2. Place the Savanah® Toilet Seat upside down on a flat surface and slip one of the L-shaped brackets into one of the slots in the side of the seat **(A)**. Push lightly on the bracket and turn the knob clockwise until the thread engages. Repeat with the second bracket.
3. Turn the knobs until the brackets are farther apart than the width of the toilet bowl.
4. Turn the seat over and place it on the toilet bowl with the narrow cutout at the front and the brackets near the back. Position the seat with the front lip on the seat against the outside front of the bowl.
5. Tighten both brackets until they are snug against the bowl.
6. Please note: The Savanah raised toilet seat is not compatible with all toilets (w/c) on the world market. A properly positioned and fitted Savanah Raised Toilet Seat is shown in **(B)**.
7. Regular checks of the brackets should be made, ensuring that the toilet seat is secure. Spare brackets are available if required. For item AA2112, AA2114Y, AA2116, AA2112L, AA2114L and AA2116L the code is 09 154 3305.

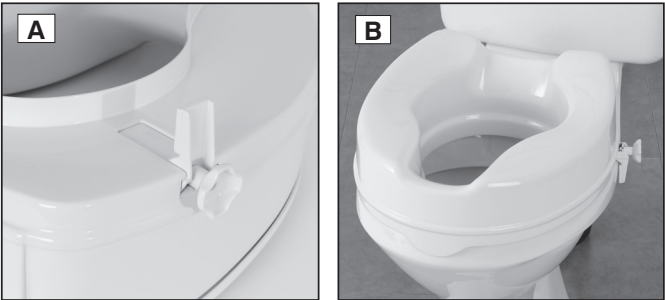

Care and cleaning

All parts can be cleaned with soap and hot water (up to 85°C). The adjusting screws can be popped out of the brackets for cleaning.

Estimated product life: 10,000 cycles.

Cautions

- The toilet seat is not to be used to support more than 190kg / 420lb / 30st
- Do not hold onto the seat as a support when getting on and off the seat
- Be sure the toilet seat is securely in place before putting weight on it
- Before each use, please ensure there is full contact between the rubber pad on the bracket and the toilet bowl
- Under no circumstances should you stand on the raised toilet seat

Assembly Instructions

1. To remove the lid, press in the hinges at the rear of the seat and gently push upwards.
2. To replace the lid, position the hinges over the rear inserts and push in until the hinges are securely in position.
3. If you do not require the lid, the lid can be replaced with caps – code 09 154 3321.

These user instructions are available in large print, please contact Patterson Medical Customer Services for a copy. Please retain these instructions for future reference.

**Manufactured for Patterson Medical Ltd.**  
Nunn Brook Road, Huthwaite,  
Sutton-in-Ashfield, Nottinghamshire,  
NG17 2HU, UK.  
**UK:** Tel: 08444 124 330 Fax: 08448 730 100  
www.pattersonmedical.co.uk  
**International:** Tel: +44 1623 448 706  
Fax: +44 1623 448 784

**Patterson Medical Products, Inc.**  
28100 Torch Parkway Suite 700,  
Warrenville, IL 60555-3938  
Tel: 1-800-323-5547 Telefax: 1-800-547-4333

**Patterson Medical Australia**  
PO Box 1006, North Ryde, NSW Australia 2113.  
Tel: 1300 473 422 Fax: 1300 766 473  
**Patterson Medical France**  
Zone Industrielle de Montjoly,  
11 Avenue de l'Industrie - CS 40712  
08013 Charleville-Mézières Cedex, France.  
Tel: 03 24 52 61 15 Fax: 03 24 52 90 34

