

User Instructions

Ready Reacher Range

M59712	Ready Reacher – 380 mm (15")
M59724	Ready Reacher – 530 mm (21")
L36195	Ready Reacher – 660 mm (26")
L36201	Ready Reacher – 813 mm (32")

General Safety Warning

Please ensure these instructions are fully read and understood before installation/ use. Retain these instructions in a safe place for future reference.

1.0 Intended Use

The Ready Reacher is to be used to pick up objects indoors or outdoors without the need for bending or rising. It is ideal for those with poor grip or arthritic hands.

2.0 Safety Precautions

- Do not pick up heavy items.
- Do not pick up hot items.
- Ensure objects are firmly held before attempting to lift.
- Store product in a safe place after use.

3.0 Use

- Position the open jaws around the object.
- Squeeze the trigger handle ensure the jaws are firmly gripping the object before attempting to lift.
- Keeping a good grip, lift the object.
- Release the trigger gently when the object is in the desired location.
- The Reacher is also supplied with a clip for storing on a walking frame.

4.0 Care & Maintenance

- Check for any wear or damage.
- Check jaws are operating correctly.
- If in doubt do not use the product.



5.0 Cleaning

- Clean with neutral detergent and warm water.
- Dry thoroughly before use.

6.0 Product Guarantee

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS Healthcare,

Sherwood House, Cartwright Way
Forest Business Park, Bardon Hill
Coalville, Leicestershire, LE67 1UB
United Kingdom.

Tel. Customer Services: +44 (0)345 121 8111

Email: customerservice@nrshealthcare.co.uk