

USER INSTRUCTIONS

ECONOMY SHOWER CHAIR RANGE

L99399	Adjustable Height Shower Chair	L99405	Fixed Height Mobile Shower Chair
M00754	Adjustable Height Mobile Shower Chair		

⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

Maximum User Weight



1. INTENDED USE

For use in a bathroom or shower, to provide a stable seated surface for clients with reduced mobility who are unable to stand during showering.



M00754 Shown

2. SAFETY

Warning: The maximum user weight must not be more than 165 kg (26 st).

Ensure seat is correctly fitted and secure.

Do not use if feet or castors are damaged.

Ensure all castors are locked when the seat is in the desired position and if leaving the client temporarily.

3. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

4. ASSEMBLY/INSTALLATION

Where the seat is supplied separately, fit to the frame by first clipping it onto the rear of the two closely spaced cross bars, and then clipping to the front cross bar.

On height adjustable models, adjust the legs by sliding to the desired height and secure with the fixing E-clips.

Set all legs to the same height.

Note: Check the seat is level and the product is stable before use

5. MAINTENANCE

Inspect the product regularly for signs of wear or damage, pay particular attention to the condition of the ferrules and castors.

Check that all fixing clips are inserted correctly and there is no excessive movement.

Check for general wear or damage.

If you see or suspect a fault do not use – contact your dealer immediately

6. CLEANING INSTRUCTIONS.

Clean regularly, use a general purpose, neutral detergent and warm water

Please check the suitability of the product before use. Do not use abrasive cleaners or cloths as this will permanently damage the surface and invalidate any warranty.

Do not use any form of solvent as this will damage the surface and weaken the product and again invalidate any warranty.

To disinfect, wipe with a dilute bleach solution, and rinse thoroughly with fresh water.

These seats may be autoclaved on cleaning cycles up to 73°C.

Dry the seat thoroughly before use.

7. ACCESSORIES & SPARE PARTS

M95583 Replacement Seat.

For more details please visit our website or call Customer Services. See end of this document.

8. PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

9. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS HEALTHCARE Sherwood House, Cartwright Way, Forest Business Park
Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services: +44 (0)345 121 8111

Email: customerservice@nrshealthcare.co.uk

NRS Healthcare is a leading supplier of community equipment and daily living aids.
Please visit one of our websites to view our full range of products:
www.healthcarepro.co.uk or www.completECAshop.co.uk