

PRODUCT INSTRUCTION SHEET



REFERENCE Birstall Bariatric Mobile Commode (195)

DESCRIPTION

Extremely robust dip-coated (non-peel) all welded frame with cellular moulded (anti-bacterial) backrest and over seat. Features include a clip-on/clip-off moulded seat and 14 litre pan with lid. Fixed height and detachable arms for lateral transfer. Fitted with 4 x 125mm diameter brake castors.

TECHNICAL SPECIFICATIONS

Seat Height (floor to top of commode seat): 500mm

Width between arms: 605mm

Seat depth: 430mm

Seat to top of arms: 180mm

Seat to top of back: 420mm

Orifice width: 250mm

Orifice depth: 280mm

Footprint width: 630mm

Footprint depth: 550mm

Maximum user weight limit: 203kg (32st)



USAGE REQUIREMENTS

1. Use on level surface only, do not exceed user weight of 203kg (32st)
2. Ensure that detachable side arms are placed securely into the tube inserts on either side of the toilet commode seat, and are secured with the pin (Eclips) provided.
3. Check that the castors are securely fitted into the base of the frame before, during and after use.
4. The product is fitted with 4 brake castors. Ensure the brake levers are fully depressed and lock the castors whilst in use.
5. The product is not designed to be “swung on” in a backward or forward motion. All 4 castors must be in contact with the floor at all times.
6. The product is designed to withstand temperatures created by the latest disinfecting commercial washing machines. Otherwise use a soft cloth dipped in neutral detergent and warm water. Do not use harsh abrasive cleaners. To disinfect clean the frame and wipe over with a solution of 1000 parts per million chlorine disinfectant – rinse and dry.
7. **DO NOT USE BLEACH BASED PRODUCTS ON THE PU UPHOLSTERY AS THIS WILL LEAD TO DISCOLORATION**