

L97718

Economy Shower Stool

General Safety Warning

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

Maximum User Weight



1. INTENDED USE

For use in a bathroom or shower, to provide a stable seated surface for users with reduced mobility who are unable to stand during showering.



2. SAFETY

Warning: The maximum user weight must not be more than 136 kg (21 st).

Inspect at regular intervals for wear and damage.

Do not use if rubber feet are damaged.

3. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

3.1 Contents

1. Bottom Leg 1 x 1
2. Bottom Leg 2 x 1
3. Long Screw x 5
4. Cupped Washer x 5
5. Seat x 1

4. ASSEMBLY/INSTALLATION

Tools required: Cross Slot Screwdriver

1. Put seat upside down on floor/flat surface.
2. Align seat legs on the seat according to the numbers indicated on the legs and seat.
3. Install mounting screws through the cupped washer, top leg and bottom leg and tighten to the seat. Check seat is secure and stable.
4. Adjust to the correct height by depressing the snap buttons and sliding the extension legs up or down the frame tube until it snaps into the appropriate adjustment hole.
5. Set all four leg extensions to the same adjustment position to ensure a level seat height. Make sure all snap buttons are fully visible and protruding.

5. IN USE

Put in shower. Make sure all four legs are level and in contact with the shower floor.

Sit centrally on the shower stool.

6. CLEANING INSTRUCTIONS

Clean regularly, wiping down with mild detergent / disinfectant solution.

7. PRODUCT DISPOSAL

Thoroughly clean/sterilise this product prior to disposal.

This product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

8. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS HEALTHCARE Sherwood House, Cartwright Way, Forest Business Park
Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services: +44 (0)345 121 8111

Email: customerservice@nrshealthcare.co.uk

NRS Healthcare is a leading supplier of community equipment and daily living aids.
Please visit one of our websites to view our full range of products:
www.healthcarepro.co.uk or www.completecareshop.co.uk