

USER INSTRUCTIONS

BARDON TROLLEY

L34587

Bardon Trolley

⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

Maximum User Weight



1. INTENDED USE

The Bardon Trolley is a versatile, easy to manoeuvre, safe and durable trolley. It is ideal to carry household items around the home. It is height adjustable and features strong plastic trays with raised edges to prevent spillage. The bottom shelf are set forward to make walking with the trolley easier.



2. SAFETY

Warning: The maximum user weight must not be more than 140Kg (22st).

The trolley is not designed to be sat upon.

Although the trolley is used when walking - it should not be used as a walking aid.

All castors must be in contact with the floor at all times during use.

It is recommended that the trolley is used on level surfaces only.

Maximum Tray Load: 5kg.

3. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

4. ASSEMBLY/INSTALLATION

Check all castors are securely fitted before and after use. The nylon bush on the top of the castor must be pushed fully into the extension leg, with no gap visible.

When adjusting the legs ensure that there are no gaps between the "e" clips and the frame. The clip spindle must be fully inserted right through the outside frame.

Ensure all Legs are the same height.

The Plastic Trays on the Bardon Trolley are designed to be a clip on/off fit for cleaning purposes.

To fit the tray, push firmly down. Make sure the trays are level with the framework.

To remove push upwards from the underside.

5. MAINTENANCE

Check all fixings are secure, before and after use.

Check the product monthly for signs of wear and tear.

Pay particular attention to the castors

6. CLEANING INSTRUCTIONS.

Clean regularly, with a mild detergent/cream cleaner and warm water.

Allow to dry thoroughly.

Do not use an abrasive detergent, cleaner or cloth to clean or dry this product.

7. ACCESSORIES & SPARE PARTS

L64270 Bardon Trolley Tray.

For more details please visit our website:

www.healthcarepro.co.uk or call Customer Services.

8. PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from plastic and metal and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

9. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 12 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover; abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS HEALTHCARE Sherwood House, Cartwright Way, Forest Business Park
Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services: 0345 121 8111

Email: customerservice@nrshealthcare.co.uk

NRS Healthcare is a leading supplier of community equipment and daily living aids.
Please visit one of our websites to view our full range of products:
www.healthcarepro.co.uk or www.completECAshop.co.uk