

User Instructions

N43504 Atlas 2 Transfer Disc

General safety Warning

Please ensure these instructions are fully read and understood. Failure to do so may result in injury. All aids for Moving & Handling should be used in accordance with local Moving & Handling recommendations. Please retain these instructions in a safe place for future reference.

1.0 Intended Use

The Atlas 2 Transfer Disc is designed to provide assistance when completing standing and seat to seat transfers. It can also be used to encourage mobility in rehabilitation.

The user must be able to support their own weight and maintain balance. They must also be able to grasp the handles and understand basic instructions.

2.0 Safety Precautions

Do not use the Atlas 2 Transfer Disc as a means of transporting users.

Do not leave the user unattended when using the product.

The carer must act as a counterbalance when the user is rising and lowering. Consideration should be given to whether the carer's body weight is sufficient to counterbalance the user's.

Do not exceed the maximum user weight; 30st (190kg).

3.0 Use

1. Position the user in close proximity to the surface to which they wish to transfer to, see figure 2.
2. Position the Atlas 2 as close as possible to the seated user. The handle bar should angle away from the user.
3. If necessary, adjust the height of the handle bar by using the hand wheel and locating holes on the rear of the upright.

Ensure the hand wheel is correctly located and secure before commencing the transfer.

Maximum User Weight

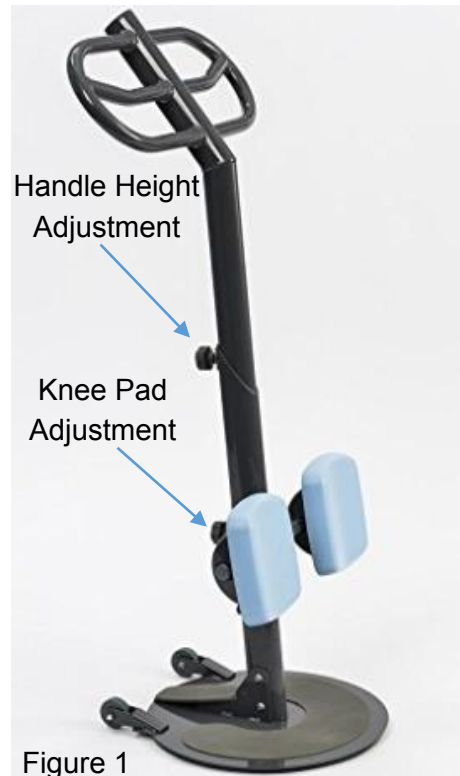


Figure 1

4. Allow the user to place their feet on the platform, ensure they are central and ideally hip width apart.
5. If necessary, adjust the height of the knee pads by using the hand wheel on the rear of the knee pad bracket. Slide the knee pad bracket to the desired height and tighten the hand wheel to secure in position.

Please Note: *The top of the knee pad should sit just below the knee cap of the user.*

Ensure the hand wheel and knee pad bracket are secure before commencing the transfer.

6. Stand behind the aid, facing the user, with one foot on the raised footplate and hold the handles. This is to counterbalance the product whilst the user uses the handle to support themselves as they rise.

7. Once the user is standing in a secure and comfortable position, rotate and manoeuvre the aid to position the user in front of the transfer surface.

The carer must maintain a firm grip on the handle bar while the user is standing on the aid.

8. Place foot back on the raised footplate and ensure a firm hold of the aid before instructing the user to lower onto the new surface.
9. Once the user is positioned on the new surface, allow them to lift their feet off the transfer disc and manoeuvre it out of the way.



Figure 2

4.0 Cleaning

Clean regularly using a general purpose, neutral detergent and warm water.

Please check the suitability of the cleaning product before use. Do not use abrasive cleaners or cloths as this will permanently damage the surface and invalidate any warranty.

NRS Healthcare

Sherwood House, Cartwright Way,
Forest Business park, Bardon Hill,
Coalville, Leicestershire, LE67 1UB, UK

To disinfect, wipe with a dilute bleach solution and rinse thoroughly with fresh water.

Allow product to dry thoroughly before use.

5.0 Maintenance

Inspect regularly, pay particular attention to the condition of the swivel platform.

Check the condition of the knee pads and handle, ensure the hand wheels are correctly located and secure.

Periodically check the product for signs of wear. Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

6.0 Product Disposal

Product should be thoroughly cleaned prior to disposal.

Product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

7.0 Product Guarantee

NRS Healthcare, guarantee this product for a period of 12 months from the date of purchase against faulty materials and workmanship. This guarantee does not cover; abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.



Tel. Head Office: 0330 100 0098

Tel. Customer Services: 0345 121 8111

Email: customerservices@nrshealthcare.co.uk

NRS Healthcare is a Leading Supplier of Community Equipment and Daily Living Aids, to view our full range, visit our website:

www.nrshealthcare.co.uk