


Bed Rope Ladder

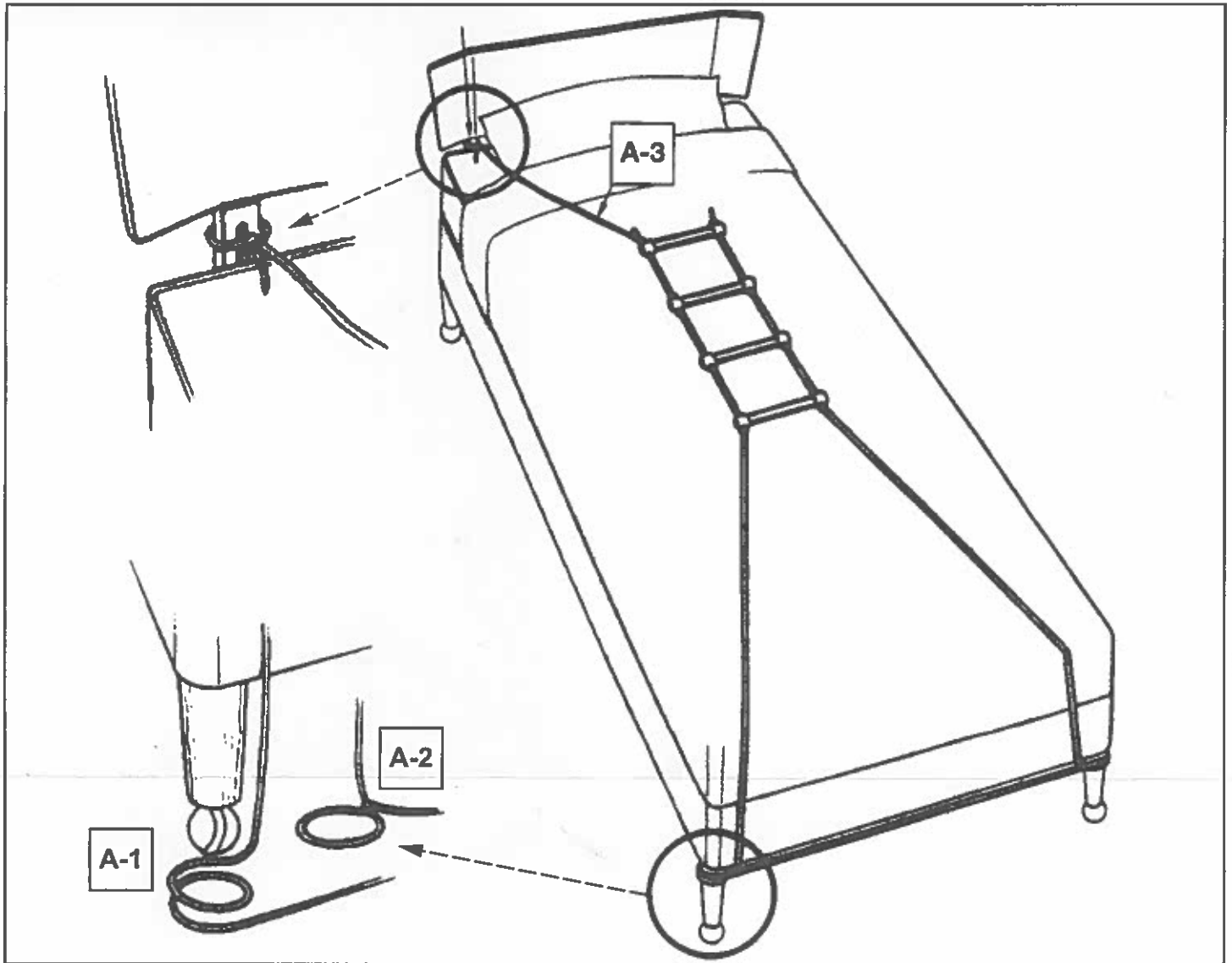

Please keep these instructions for future reference.

INTENDED USES

- To compensate for a disability by helping a person rise to a sitting position from a lying position.

INSTRUCTIONS FOR USE

The rope ladder hoist can be used on single and double beds that have legs.

1. Attach the rope around the legs at the foot of the bed as shown in (A). Use one of the

following methods to attach the rope:

- Form turns in the rope and slip them under the legs of the bed (A-1).
- OR
- Wrap the rope around the leg and twist it around itself (A-2).
2. The user pulls him- or herself up in bed by pulling on successive rungs of the ladder with a hand-over-hand movement.
 3. A string (not included) can be attached to the top rung of the ladder and the

headboard to keep the ladder ready to use (A-3).

CARE AND CLEANING

Wipe with a damp cloth and detergent solution.

CAUTIONS

- Inspect the rope ladder prior to each use for signs of wear. Do not use the rope ladder if it appears weakened.
- Be sure the rope ladder is securely attached to the bed before each use.

DEUTSCH

Strickleiter-Hebezug

Bitte bewahren Sie diese Anleitungen zum späteren Nachlesen gut auf.

BESCHREIBUNG

- Aufsetzhilfe aus einer Liegeposition für behinderte Personen

GEBRAUCHSANLEITUNG

Die Strickleiter eignet sich für Einzel- und Doppelbetten mit Beinen.

1. Befestigen Sie das Seil um die Beine am Fuß des Betts (siehe Abb. A). Verwenden Sie eine der folgenden Methoden, um das Seil zu befestigen:

- Bilden Sie Schlaufen mit dem Seil, und schieben Sie sie unter die Beine des Betts (A-1).

ODER

- Wickeln Sie das Seil um das Bein, und drehen sie es um sich selbst (A-2).
2. Der Benutzer zieht sich selbst im Bett hoch, indem er eine Leitersprosse nach der anderen mit einer übergreifenden Handbewegung erfasst.
 3. An der obersten Sprosse der Leiter und dem Kopfende des Betts kann ein Strick (nicht

mitgeliefert) befestigt werden, um die Leiter einsatzbereit zu halten (A-3).

PFLEGE UND REIEIGUNG

Mit einem feuchten Tuch und Spülmittellösung abwischen.

VORSICHTSMAßNAHMEN

- Inspizieren Sie die Strickleiter vor jedem Gebrauch auf Anzeichen von Abnutzung. Verwenden Sie die Strickleiter nicht, wenn sie Schwachstellen aufweist.
- Prüfen Sie vor jedem Gebrauch, dass die Strickleiter sicher am Bett befestigt ist.