



JLP125PI

ADJUSTABLE CHAIR RAISER

Welcome to your new Gordon Ellis product.
We hope that it proves a useful aid in your daily life.



EN ANY SERIOUS ACCIDENT OCCURRING THROUGH USE SHOULD BE REPORTED TO THE EC REP (EU CUSTOMERS) OR THE MANUFACTURER (UK & NON-EU CUSTOMERS) - CONTACT DETAILS OVER LEAF




SEE REVERSE FOR FITTING INSTRUCTIONS
AVAILABLE IN THE FOLLOWING LANGUAGES:
EN




MAXIMUM USER WEIGHT
PER SET OF FOUR


Your JLP125PI ADJUSTABLE CHAIR RAISER was specifically developed by our UK Design Team with the intention of compensating for an injury, or overcoming a temporary or permanent disability. Hopefully making daily living that little bit easier.

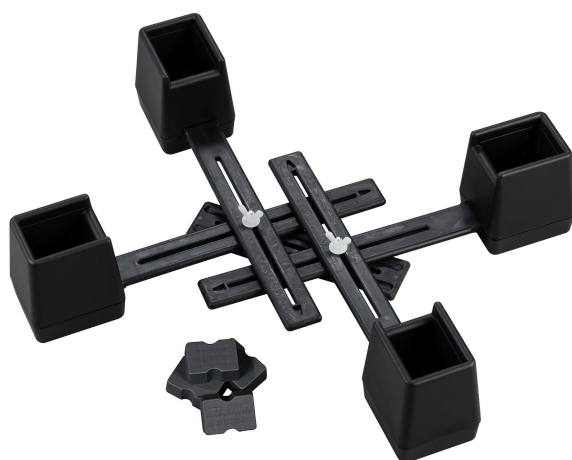

MAXIMUM CHAIR FOOTPRINT

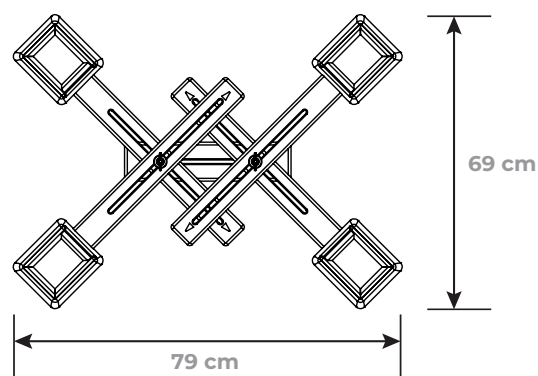

RAISING HEIGHT

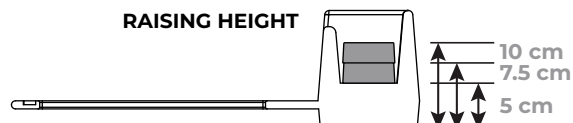

MAXIMUM CHAIR LEG SIZE = 6.4CM X 7.6CM

LAN & HAM

INSPIRED BY YOUR NEEDS



GORDON ELLIS & CO.
TRENT LANE
CASTLE DONINGTON
DERBY. ENGLAND DE74 2AT

EC REP

AUTHORISED REPRESENTATIVE
ADVENA LTD, TOWER BUSINESS CENTRE, 2ND
FLOOR, TOWER STREET, SWATAR, BKR 4013
MALTA



MD

IFU-REF LP/JLP125PI/T120-1

PART REF LP/JLP125PI

VIEW THE ENTIRE RANGE ONLINE AT
GORDONELLISHEALTHCARE.CO.UK

gordonellis.healthcare
 @EllisHealthcare











INSTRUCTIONS FOR USE

i EN TRIED AND TESTED BY OCCUPATIONAL THERAPISTS FOR DECADES, THE LANGHAM ADJUSTABLE LINKED CHAIR RAISER IS DESIGNED FOR FIRESIDE CHAIRS, TO GIVE THE HIGHEST LEVEL OF STABILITY. THE UNIT COMES FULLY ASSEMBLED IN AN EASY TO FIT PACK, REQUIRING THE USER TO SIMPLY ADJUST TO THE SPECIFIC CHAIR BEING RAISED.

FITTING INSTRUCTIONS:

1. PLACE THE CHAIR RAISER ON THE FLOOR BEHIND THE CHAIR WITH THE SPREADER BAR RUNNING SIDE TO SIDE.
2. LOOSEN THE WINGNUTS.
3. ADJUST TO SIZE SO THAT THE CHAIR LEGS WILL EASILY DROP INTO THE RAISER UNITS.
4. TIP THE CHAIR FORWARDS AND PLACE THE CHAIR LEGS INTO THE REAR RAISER UNITS. NOTE: THE FURNITURE LEG OR CASTOR MUST REST ON THE INNER BASE OF THE RAISER UNIT ONLY.
5. MOVE TO THE FRONT OF THE CHAIR AND TIP IT BACKWARDS. BRING THE FRONT RAISER UNITS FORWARD UNTIL THEY ARE UNDER THE FRONT LEGS. LOWER THE CHAIR.
6. PUSH THE RAISER UNITS TIGHT AGAINST EACH OF THE CHAIR LEGS AND THEN ADJUST THE RAILS UNDERNEATH THE CHAIR SO THAT THEY ARE AS CENTRAL AS POSSIBLE TO REMOVE TRIP HAZARDS. THE SPREADER BAR SHOULD STILL RUN SIDE TO SIDE.
7. TIGHTEN ALL OF THE WINGNUTS.
8. RAISER BLOCKS CAN BE INSERTED AS NECESSARY TO FURTHER RAISE BY 24MM OR 48MM - MAXIMUM TWO PER RAISER UNIT. ENSURE EACH RAISER UNIT IS EQUAL IF EXTRA HEIGHT IS ADDED.
9. ENSURE THE CHAIR AND RAISER UNITS ARE STABLE BEFORE EACH USE AND IF KNOCKED.

! DEVICE TO BE USED IN STATIC MODE ONLY. EXTRA CARE SHOULD BE TAKEN TO AVOID SUDDEN OR UNDUE MOVEMENT. IT IS THE RESPONSIBILITY OF THE OWNER/ USER TO CARRY OUT AN INITIAL AND REGULAR LOCAL RISK ASSESSMENTS. DO NOT USE WITH ANY OTHER DEVICE UNLESS STATED. TAKE CARE TO AVOID ENTRAPMENT OF FINGERS IN ADJUSTMENT SLOTS.

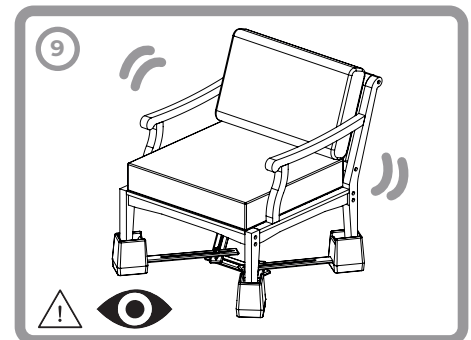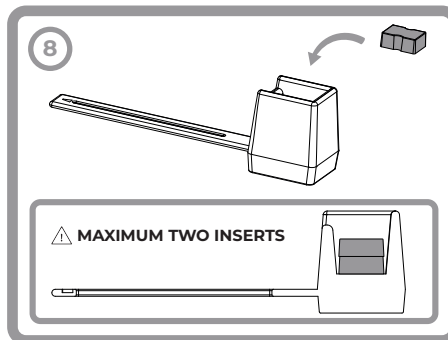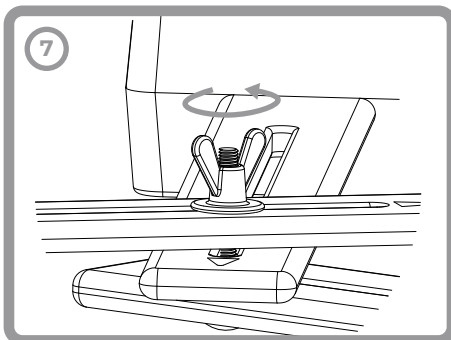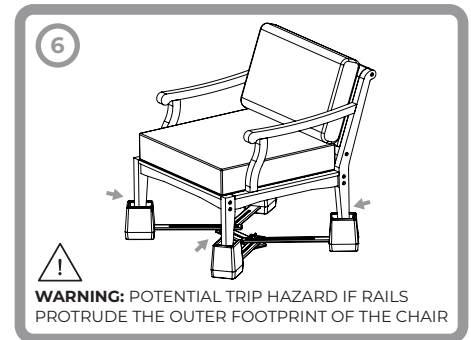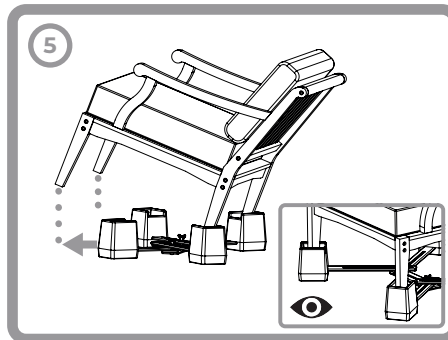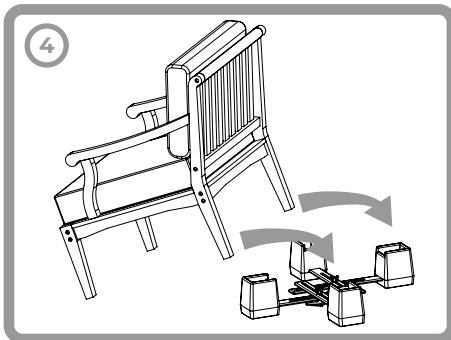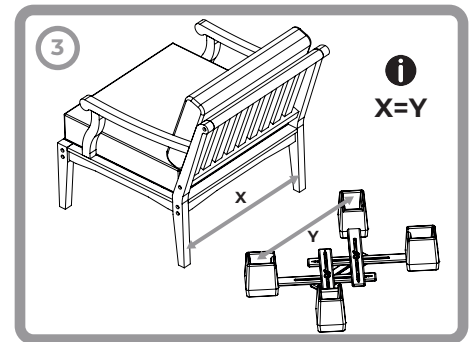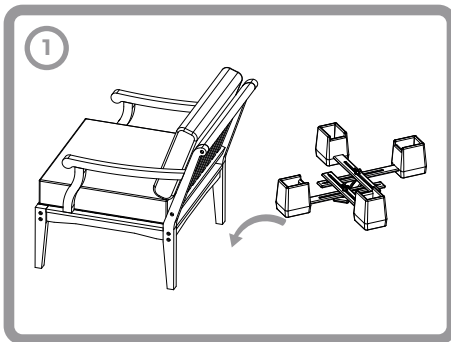

PH 5 - 9

CLEANING INSTRUCTIONS

WIPE DOWN WITH A NON-ABRASIVE CLOTH AND SOAPY WATER. DEVICE IS FULLY AUTOCLAVABLE.

VIEW THE ENTIRE RANGE ONLINE AT
GORDONELLISHEALTHCARE.CO.UK

gordonellis.healthcare
 @EllisHealthcare

UK
CA