

## User Instructions

### Wooden Transfer Board Range

<b>F20421</b>	<b>Short Transfer Board (600mm)</b>	<b>F20433</b>	<b>Long Transfer Board (760mm)</b>
<b>F20998</b>	<b>Curved Transfer Board</b>	<b>P17889</b>	<b>Snake Transfer Board</b>
<b>M01175</b>	<b>Lightweight Curved Board</b>	<b>L62170</b>	<b>Bariatric Curved Transfer Board</b>

#### General Safety Warning

Please ensure these instructions are fully read and implemented. Failure to do so may result in injury. Retain these instructions in a safe place for future reference.

Maximum User Weight



Maximum User Weight  
L62170 Only



#### 1.0 Intended Use

These Transfer Boards are designed to assist users transferring between two seated surfaces of similar height eg. wheelchair to bed.

#### 2.0 Safety Precautions

Do not exceed the maximum user weight of the product, 30st (190kg) except L62170 40st (254kg).

Do not attempt to transfer across distances greater than the board allows. Ensure both ends of the board sufficiently overlap the transfer surfaces. NRS Healthcare recommend a minimum of 150mm.

Do not attempt to use multiple boards for a single transfer.

Transfer surfaces should be as level as possible, avoid slopes.

#### 3.0 Unpacking

Remove all packaging and check product carefully. Do not use if there are any signs of damage and contact your supplier immediately.

#### 4.0 Material Specification

Birch Plywood with clear lacquer finish.



## 5.0 In Use Instructions

Position the transfer surfaces next to each other, at right angles as close as possible.

Place the transfer board over the gap between the two surfaces, figure 6 & 7. *Ensure it is sufficiently supported on both sides, a minimum of 150mm overlap.*

The user can now use the board to manoeuvre from one surface to the other. *Take care to ensure fingers do not become trapped under the board.*

Where possible the user should push down with their hands during transfer to lift themselves slightly to avoid rubbing on the board. This is particularly important where there is a loss of sensation. Assistance from a carer may be required in cases of severe limitation.



**Figure 6 – P17889 in use example**



**Figure 7 – M01175 in use example**

## 5.0 Cleaning

Clean regularly using a general purpose, neutral detergent and warm water.

Please check the suitability of the cleaning products before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.

## 6.0 Maintenance

Inspect the product regularly for any signs of wear or damage to the surface of the board which may cause friction and damage the skin.

Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

## 7.0 Product Disposal

Product should be thoroughly cleaned prior to disposal.

Product is manufactured from wood composite and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

## 8.0 Product Guarantee

NRS Healthcare guarantee this product for a period of 12 months from the date of purchase against faulty materials and workmanship. This guarantee does not cover; abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

### NRS Healthcare

Sherwood House, Cartwright Way,  
Forest Business park, Bardon Hill,  
Coalville, Leicestershire, LE67 1UB, UK

Tel. Head Office: 0330 100 0098

Tel. Customer Services: 0345 121 8111

Email: [customerservices@nrshealthcare.co.uk](mailto:customerservices@nrshealthcare.co.uk)



**NRS Healthcare** is a Leading Supplier of Community Equipment and Daily Living Aids, to view our full range, visit our website:

**[www.nrshealthcare.co.uk](http://www.nrshealthcare.co.uk)**