

P23075

SPA Shower Stool

⚠ GENERAL SAFETY WARNING ⚠

Please make sure that you read and follow these instructions carefully. Failure to do so can result in injury. Keep these instructions in a safe place for future reference.

INTENDED USE

The SPA Shower Stool has been designed to provide users with a stable seated support for use during showering. The Shower Stool is suitable for users with reduced mobility, who are unable to stand up for long periods of time.



SAFETY

The maximum user weight must not be greater than 20st / 127kg.

Make sure that all legs are in contact with the shower pan, to avoid rocking and instability.

Do not stand on the stool under any circumstances, failure to comply can result in serious injury.

Do not lean backwards on the stool, it can become unstable and fall.

Wipe dry after each use.

MAXIMUM USER WEIGHT



UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged.

ASSEMBLY

The product is supplied fully assembled.

CLEANING

Clean regularly using a general purpose, neutral detergent and warm water.

Please check the suitability of the cleaning products before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.

To disinfect, wipe with a dilute bleach solution and rinse thoroughly with fresh water.

Allow the product to dry fully prior to use/storage.

MAINTENANCE

Inspect the product regularly for signs of wear or damage, pay particular attention to the condition of the ferrules.

Any suspected faults or damage should be immediately reported to the supplier and the product taken out of use.

ACCESSORIES AND SPARES

P57875: Replacement set of ferrules.

P57851: Replacement set of 10 x non-slip strips.

PRODUCT DISPOSAL

Product should be thoroughly cleaned prior to disposal.

Product is manufactured from metal, plastic and bamboo and can be disposed of through Local Authority Household Waste and Recycling Centres or through local Trade Waste Collectors.

PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 12 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover; abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

OTHER NRS PRODUCT IN THE SPA RANGE

P23099 - SPA Bath Step



P42409 - SPA Perching Stool



NRS HEALTHCARE Sherwood House, Cartwright Way, Forest Business Park, Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.
Tel. Customer Services: 0345 121 8111
Email: customerservices@nrshealthcare.co.uk

NRS Healthcare is a leading supplier of community equipment and daily living aids, to view our full range, visit our website:

www.nrshealthcare.co.uk

P23075-UI-2