

# User Instructions

## Shelford Support Frame G21789 - Right Hand G21791 - Left Hand

Maximum User Weight



### General Safety Warning

Please ensure these instructions are fully read and understood before installation/use. Retain these instructions in a safe place for future reference.

### Intended Use

This multi purpose support frame can be used individually, as a bath rail or toilet support, or in pairs on either side of the toilet to aid the user to sit or stand with greater ease and stability.

### Safety Precautions

- Check the product for signs of damage prior to fitting, report any defects to your supplier or NRS immediately.
- It is recommended that this product should only be installed by a suitably trained and qualified person.
- Screws and fixings are not supplied with this product, it is recommended a survey of the floor/ flooring materials is carried out to ensure the correct type of fixings are used.
- Note it is recommended in a wet environment that stainless steel screws are used to avoid corrosion.
- For hygiene and safety purposes check the product regularly for any signs of wear or damage, paying particular attention to the surface coating
- Check regularly that all screws and fixings are secure and the frame is stable
- Due to the nature of this product, the large range of floor types and different operating environments to which the product may be installed it is not possible to specify a maximum user weight.
- Note: This product should only be used for it's intended purpose



### Fitting Instructions:

- Position the frame/frames in the desired position.
- Ensure the floor is of suitable construction
- Mark through the holes in the frame
- Drill and plug holes with suitable fixings
- Using suitable screws fix the frame, making sure all screws are firmly fixed in position.
- Before use check the frame is firm and stable.

### Cleaning Instructions

Clean regularly, use a general purpose, neutral detergent and warm water. Do not use an abrasive detergent, cleaner or cloth to clean or dry this product. To disinfect, wipe with a mild chloride disinfectant.

Manufactured for **NRS Healthcare**  
Sherwood House, Cartwright Way,  
Forest Business Park, Bardon Hill,  
Coalville, Leicestershire.  
LE67 1UB, UK

Tel. Main Office: 0330 100 00 98  
Tel. Customer Services: 0845 120 4522  
Email: [customerservice@nrs-uk.co.uk](mailto:customerservice@nrs-uk.co.uk)