

# USER INSTRUCTIONS

## PU PERCHING STOOL RANGE

L35555	Adjustable Height Stool	L35567	Adjustable Height Stool with Armrests
L35579	Adjustable Height Stool with Armrests & Backrest	L35580	Adjustable Height Stool with Armrests & Padded Backrest
L61049	Extra Low Adjustable Height Stool with Armrests & Padded Backrest	L61050	Extra Low Adjustable Height Stool with Padded Armrests & Backrest

### ⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

### Maximum User Weight



## 1. INTENDED USE

The perching stool range is intended to be used to provide a stable, seated support, where a user is unable to stand for long periods, the seat slopes forward to allow the user to achieve a relaxed “perching” position when sitting and also to assist when rising. This product is not for use in a shower or wet-room.



**L35580 Shown**

## 2. SAFETY

Do not exceed the maximum user weight limit.  
Inspect at regular intervals for wear and damage.  
Check seat for damage or tearing.  
Do not use if rubber feet are damaged.  
Do not use this product in a shower or wet-room.

## 3. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

## 4. HEIGHT ADJUSTMENT

To adjust the stool to the desired height, remove the plastic clips connecting the stools' legs and leg extensions.

Slide the leg extensions to the required position, ensuring that all legs are equal in height.

Replace the plastic clips ensuring that each clip is fully connected.

**Note:** Ensure the floor surface is level and even to avoid rocking.

## 5. MAINTENANCE

Inspect the product regularly for signs of wear or damage, pay particular attention to the condition of the ferrules

Check that all fixing clips are inserted correctly and there is no excessive movement.

Check for general wear or damage.

If you see or suspect a fault do not use – contact your dealer immediately

## 6. CLEANING INSTRUCTIONS.

Clean regularly using a general purpose, neutral detergent and a damp cloth.

Please check the suitability of the cleaning products before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.

To disinfect, wipe with a dilute bleach solution, and rinse thoroughly with fresh water.

Allow product to fully dry prior to use/storage.

## 7. PRODUCT DISPOSAL

This product should be thoroughly cleaned prior to disposal. This product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

## 8. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

**NRS HEALTHCARE** Sherwood House, Cartwright Way, Forest Business Park  
Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services: +44 (0)345 121 8111

Email: [customerservice@nrshealthcare.co.uk](mailto:customerservice@nrshealthcare.co.uk)

**NRS Healthcare** is a leading supplier of community equipment and daily living aids.  
Please visit one of our websites to view our full range of products:  
[www.healthcarepro.co.uk](http://www.healthcarepro.co.uk) or [www.completecareshop.co.uk](http://www.completecareshop.co.uk)