

User Instructions for YESS Range Plus Size Perching Stools.

M/X/SX – 158 / 159 / 160 / 161

Introduction

Congratulations on your purchase of a YESS Range Perching Stool from Cefndy Healthcare. Please ensure you read and understand these instructions in full before use. Please retain these for future reference.

The range is available in numerous combinations and sizes to suit your needs.

The Perching Stool Range is designed for comfort with heat sealed padded vinyl upholstery or luxury contoured polyurethane foam with pronounced mouldings to counter any slippage from the inclined position.

The whole range is height adjustable at 25mm intervals and the feet are non-slip and non-marking.

All the joints are fully welded to aid product hygiene by reducing the possibility of bacterial entrapment. The steel framework is also coated in an anti-bacterial powder coated finish and the stools are stackable.



Specifications & Pack Contents

	Mediatric Size	Bariatric	Super Bariatric
Seat Height	550 – 700mm	550 – 700mm	550 – 700mm
Seat Width Vinyl	540mm	640mm	640mm
Seat Width Polyurethane	540mm	540mm	540mm
Seat Depth	320mm	320mm	320mm
Seat to Top of Arms	160mm	160mm	160mm
Width Between Arms	600mm	700mm	800mm
Max Footprint	660 x 560mm	760 x 560mm	860 x 560mm

Pack Contents

- Perching Stool
- User Instructions
- Batch Label – This will be found on the product and will include the product code as well as a number. In the event of needing to contact the manufacturer you will need to quote this number.


User Instructions for YESS Range Plus Size Perching Stools.

M/X/SX – 158 / 159 / 160 / 161

The Perching Stool Range

The YESS perching stool range comes in many configurations and variations to suit your need. All the stools come with deeply padded vinyl upholstery as standard with an option to change this for the deluxe PU foam upholstery (smaller pictures). Where applicable stools come with a fixed back, but they all have the option for a removable back.

The pictures below all show the Bariatric size (700mm width). They are all also available in the smaller Mediatic size (600mm width) and the larger Super-Bariatric (800mm width).

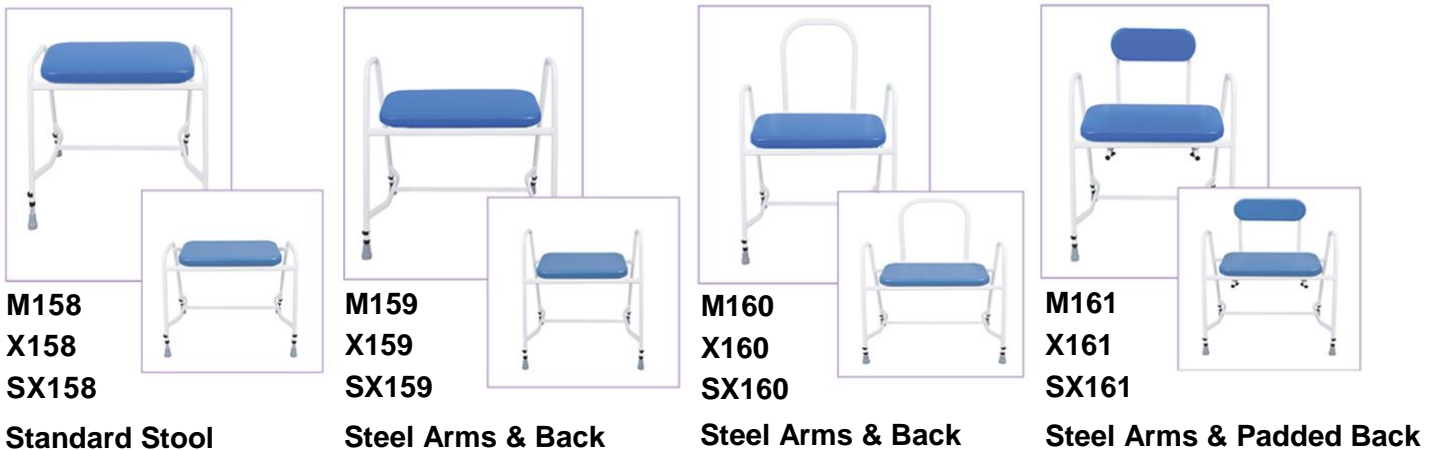

For PU upholstery add "P" to the code - For Detachable Back add "D" to the code - Add "DP" for both.

Assembly Instructions

Please note:

- When unpacking carefully examine the product to ensure it is complete.
- Before use check for any damage. If damaged please contact your supplier and refrain from using the product.
- **Do not attempt to use if the product is damaged.**

Correct Use – Height Adjustment

- Remove the pin clip from each of the legs
- Select the required height of the frame by gently sliding each extension leg in or out.
- When desired height is found refit the pin clip into the appropriate hole in each extension leg
- Ensuring it goes fully through. Ensure that all legs are set to the same height
- This product is designed so the seat will be lower at the front than the rear (see warning section)

Cleaning

- Use mild detergent with warm water to wipe down framework on at least a weekly basis more frequently if required.
- Do not use an abrasive cleaner as this may damage the frame and invalidate your warranty.

Use of power/jet washing & industrial washing machines

- *It is not recommended that polyurethane is cleaned using power/jet washing or industrial washing machines*
- If either of the above is used to clean equipment care must be taken since high-pressure water can be penetrative.
- Care should be taken that frames are not placed too close to high-pressure water jets.
- After cleaning ensure any residual water is removed from the framework as it could cause permanent rusting of the frame (leave frames open in a warm environment for several hours to ensure all internal moisture evaporates).
- We cannot offer advice on industrial washing machine

Maintenance Requirements

- The product should be checked on a regular basis for damage or wear and tear – this includes flaking or cracking paint. Any undue signs of damage should be noted and reported to the supplier in the first instance. Should damage be evident, then the user should refrain from using the product until advice has been sought from the supplier.

Warning

- **This product is designed so when in use the seat will be lower at the front than the rear, the easy way to check if the product is set for correct use is to ensure that you can see an equal number of holes visible on all four legs. The product may have been sent with the front legs lower than the rear for transportation purposes.**
- This product is intended for users up to a maximum of 285kg/45stone in weight.
- We recommend a full risk assessment is carried out before use. This should include consideration for the equipment's intended use, the capacity of the user to safely use the equipment, the environment in which the equipment is intended to be used and **all** other relevant considerations.
- It is advised that the frame is checked periodically for signs of wear and tear. Any problems please report to your supplier immediately.

Warranty

This product carries a 12 month warranty from the date of purchase against manufacturing defects, damage incurred through misuse is not covered. This warranty does not affect your statutory rights. Do not use in the event of fault or damage.

Disposal Instructions

- Appropriate hygiene standards should be applied before disposal to minimise the risk of infection.
- Materials used are non-hazardous and can be disposed of in line with local recycling policies.

Contact Details

We have attempted to cover most eventualities, however if you have any specific queries, please contact:

Cefndy Healthcare
Cefndy Road
Rhyl
Denbighshire
LL18 2HG

Tel: 0044 1745 343877

E-mail: cefndy.sales@denbighshire.gov.uk