

Product Technical Data Sheet 012 N72838 Aluminium Swivel Bath Seat

Product Range

N/A	N/A
-----	-----

Product Description & Intended Purpose

This Swivel Bath Seat is designed to enable a user of limited mobility to rotate, whilst seated, into the bath to allow seated washing.

Product Features

- Moulded polypropylene (PP) seat complete with drainage holes.
- Powder coated, rust resisting aluminium frame.
- Adjustable frame to fit the most common bath tub sizes. (It is not suitable for corner baths, baths which have uneven / profiled top edges or some baths with deeply moulded side panels.)



Technical Specification

Dimensions (mm):

- Support Frame width 660-760
- Seat Width: 460
- **Maximum User Weight: 160Kg (25st)**

Installation Information *(Note: for detailed information refer to User Instructions)*

- Slide the armrests into the locating tubes on the moulded seat.
- Slide the mounting frame into the locating tubes on the moulded seat, choosing the correct width to suit your bath.
- Place the seat on the flat bath ledges, extend the frame by removing the 'e-clips', adjust to size and replace 'e-clips'.
- Secure the frame using the clamps under the seat ensuring a secure, tight fit.

Safety Precautions

- When placing on the bath ensure all four feet of the bath seat mounting frame are resting on the edge of the bath by at least 50mm (2").
- Check the Hand wheel screws are contacting either side of the bath wall. Do not over tighten the hand wheels as this could damage the bath.
- Do not exceed the maximum user weight limit.
- Always sit in the centre of the seat and never attempt to stand on the product.

PTDS012

N72838 Aluminium Swivel Bath Seat

Cleaning

- Clean regularly, using a general purpose, neutral detergent and warm water.
- Please check the suitability of the detergent prior to use. Do not use abrasive cleaners or cloths as this will permanently damage the surface and invalidate any warranty.
- To disinfect, wipe with a dilute bleach solution, and rinse thoroughly with fresh water.
- Always allow the product to fully dry prior to stacking and storage.

Care & Maintenance

- Inspect the product regularly for signs of wear or damage, pay particular attention to the swivel and lock movement.
- Check that all fixing clips and tubes are inserted correctly and there is no excessive movement.
- Check for general wear or damage. Any damage should immediately be reported to the supplier and the product taken out of service.

Accessories / Related Products

None

FAQ's

xxx

Product Supplied By:

NRS Healthcare Sherwood House, Cartwright Way, Forest Business Park, Bardon Hill, Coalville, Leicestershire. LE67 1UB UK