

User Instructions

Mowbray Toilet Frame & Seat Extra Wide

M48805 – Mowbray Frame & Seat Extra Wide

M48817 – Mowbray Frame & Seat Extra Wide – Floor Fixed

General safety Warning

Please ensure these instructions are fully read and implemented. Failure to do so may result in injury. Retain this instruction manual in a safe place for future reference.

1.0 Intended Use

The Mowbray toilet frame and seat is designed to assist those who have difficulty in lowering onto and rising from a standard toilet. The plastic moulded seat offers support in a natural position and the frame is height adjustable.

2.0 Safety Precautions

Avoid extending the legs in a manner which causes the seat to be on a slant.

Do not exceed the maximum user weight of 40st 254kg.

Avoid using the product on a wet or slippery floor.

Do not attempt to stand on the frame or seat.

3.0 Unpacking

Remove all packaging and check all parts are present and undamaged. Any suspected faults or damage should immediately be reported to the supplier and the product taken out of action.

Spare and replacement parts are available for this product, contact customer services.

Maximum User Weight



4.0 Installation

Please note: The free standing versions of this product are delivered ready assembled. If your product is not assembled, please follow instructions.


Fit the moulded toilet seat to the frame by clipping onto the front and back cross tubes as shown in the picture above.

To adjust the height:

- Remove the 'e-clips', slide the legs to the desired height, align the holes and re-insert the 'e-clips'.

Position frame around toilet.

Model M48817 only, secure the frame to the floor by screwing down the fixing legs.

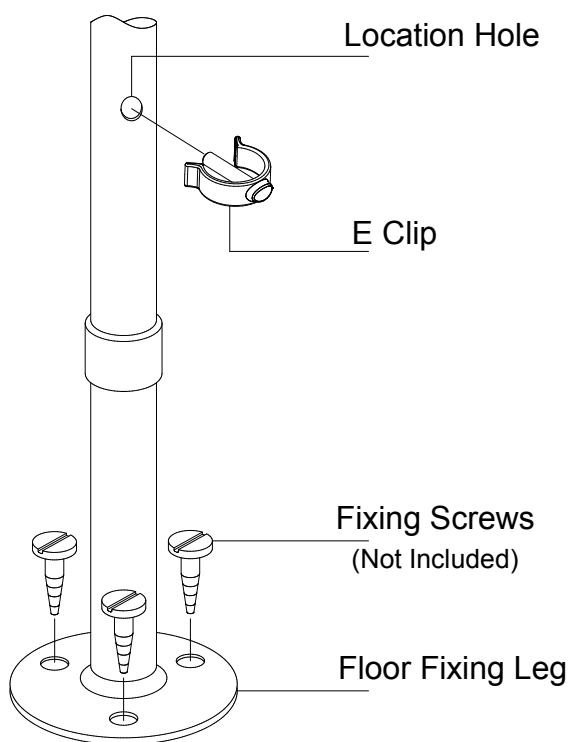

Please Note: It is important to set all four legs to the same height to prevent the frame from rocking. Height adjustment is not possible once the legs are secured to the floor.

5.0 Cleaning

Clean frame and seat regularly using a general purpose, neutral detergent and warm water.

To disinfect, wipe with a dilute bleach solution and rinse thoroughly with fresh water.

Please check the suitability of the cleaning product before use. Do not use abrasive cloths or fluids.

The seat can be removed from the frame and autoclaved on cycles up to 70°C.

NRS Healthcare

Sherwood House, Cartwright Way,
Forest Business park, Bardon Hill,
Coalville, Leicestershire, LE67 1UB, UK

6.0 Maintenance

Check that all 'e-clips' are inserted correctly and floor fixing screws are secure. There should be no excessive movement.

Regularly check the condition of the ferrules.

Periodically inspect the product for signs of general wear or damage. Report any suspected faults to supplier and remove the product from service.

7.0 Product Disposal

Product should be thoroughly cleaned prior to disposal.

Product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

8.0 Product Guarantee

NRS Healthcare, guarantee this product for a period of 12 months from the date of purchase against faulty materials and workmanship. This guarantee does not cover; abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.



Tel. Head Office: 0330 100 0098

Tel. Customer Services: 0345 121 8111

Email: customerservices@nrshealthcare.co.uk

NRS Healthcare is a Leading Supplier of Community Equipment and Daily Living Aids, to view our full range, visit our website:

www.nrshealthcare.co.uk