

L70294

NUVO Extra Wide Toilet Frame

General Safety Warning

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

1. INTENDED USE

The Nuvo Toilet Frame range is designed to assist those who have difficulty in lowering onto, or rising from a standard toilet. They can also be used in conjunction with a raised toilet seat.



2. SAFETY

Warning: The maximum user weight must not be more than 254Kg (40st).

Inspect product at regular intervals for signs of wear or damage.

Make sure the floor surface is level and even to avoid rocking.

Do not use if rubber feet are damaged.

3. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

4. IN USE

4.1 Stool Height Adjustment

Remove the plastic e-clips connecting the frame legs and leg extensions.

Slide the leg extensions to the required position.

Note: Make sure that all legs have been set to the same height.

Maximum User Weight



Replace the plastic e-clips making sure that each e-clip is fully connected.

5. MAINTENANCE

Check that all fixing e-clips are inserted correctly.

Check for general wear or damage.

If you see or suspect a fault do not use – contact your dealer immediately.

6. CLEANING INSTRUCTIONS.

Clean the frame and armrests regularly, use a general purpose, neutral detergent and warm water.

Do not use an abrasive detergent, cleaner or cloth to clean or dry this product.

To disinfect, wipe with a mild chloride disinfectant (solution of 1000 parts per million).

Dry thoroughly before use.

7. PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

8. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 12 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover; abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS HEALTHCARE Sherwood House, Cartwright Way, Forest Business Park
Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services: 0345 121 8111

Email: customerservice@nrshealthcare.co.uk

NRS Healthcare is a leading supplier of community equipment and daily living aids.
Please visit one of our websites to view our full range of products:
www.healthcarepro.co.uk or www.completecareshop.co.uk