Uplifting Seat

Instruction Manual for LA020

Dear customer,

Your new uplifting seat offers optimum functionality and comfort.

Read these instructions carefully before assembling and using the product! Retain the instructions for future reference! If you pass the product on, please also pass these instructions. The manufacturer and importer assume no liability for incorrect use of the product or non-observance of the information in these instructions!

Intended use

The uplifting seat should only be used to assist sitting down and standing up from a chair. It is designed for use on a firm chair, such as a dining chair or similar. The product is suitable for indoor and private use only. The product is unsuitable for trade use.

Technical Data:


Max load:	105kg
Size:	54 x 45mm
Weight:	3450g

To Use - Instruction

1. The up-lifting seat has 6 grades of weight adjustments.
2. Adjust the weight capacity according to body weight. Snap the spring into hooks.
3. Place the seat in the chair after adjusting in a correct grade. The seat will go down when the person sits on the seat with the suitable weight capacity.
4. The seat will help people stand up easier. Make sure people hold the armrest of the chair when using the seat.

Safety instructions

- Only use the up-lifting seat for the intended use.
- Only use the up-lifting seat in the chair with armrest.
- Do not hold the edge of the seat when sitting down because the seat may clamp your hand!
- Do not stretch the hand into the gap of the seat when people sit down on it.
- Do not sit down when people adjusting the weight capacity.
- Do not make any modifications to the product and do not repair it yourself!
- Only obtain spare parts from the manufacturer or the wholesaler. Incorrect repairs can cause unforeseeable damage!
- Keep the packing material away from children and pets (risk of suffocation).
- Ensure that another person is close by when using the seat. This person can help you with sitting down or standing up if necessary.
- Do not use this product with visible or suspected damage!

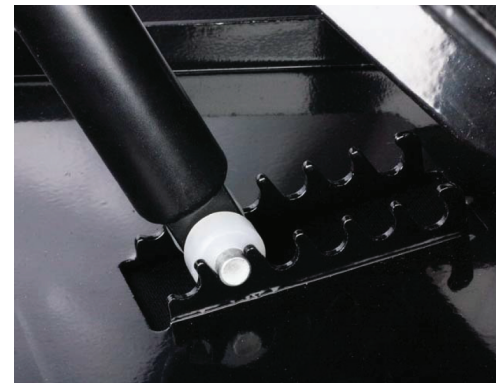Cleaning

Caution

Do not use any caustic or abrasive cleaning agents or sponges. These can damage the surface.

Disposal

- Disposal of the packing in an environmentally friendly manner.
- Dispose of it were valuable waste is collected.
- Dispose of the product through an approved disposal company or the municipal disposal facilities.
- Observe the current regulations.
- If in doubt, consult your local authority about an environmentally friendly disposal.

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