

User Instructions - N72838

NRS Aluminium Swivel Bath Seat

General Safety Warning

Please ensure these instructions are fully read and implemented. Failure to do so may result in injury to the user. Retain in a safe place for future reference.

Intended Use

This Swivel Bath Seat is designed to enable a user of limited mobility to rotate, whilst seated, into the bath to allow seated washing. It is suitable for most standard baths but is not suitable for corner baths, baths which have uneven / profiled top edges or some baths with deeply moulded side panels. The frame can be adjusted to fit most common bath sizes.

Specification

Support Frame width	660-760mm
Seat Width:	460mm
Max User Weight:	160 Kg (25st)

Safety Precautions

- Check all fixings are installed correctly
- When placing on the bath ensure all four feet of the bath seat mounting frame are resting on the edge of the bath by at least 50mm (2")
- Check the Handwheel screws are contacting either side of the bath wall. Do not over tighten the handwheels as this could damage the bath.
- Do not exceed the maximum user weight limit.
- Always sit in the centre of the seat
- Do not lean on the edges of the seat
- Do not stand on the seat or any other part of the product
- Inspect at regular intervals, see care and maintenance section



Contents List

1 - Seat Assembly	1
2 - Armrest	2
3 - Mounting Frame	2
4 - Fixing e-clips	4
User Instruction	1

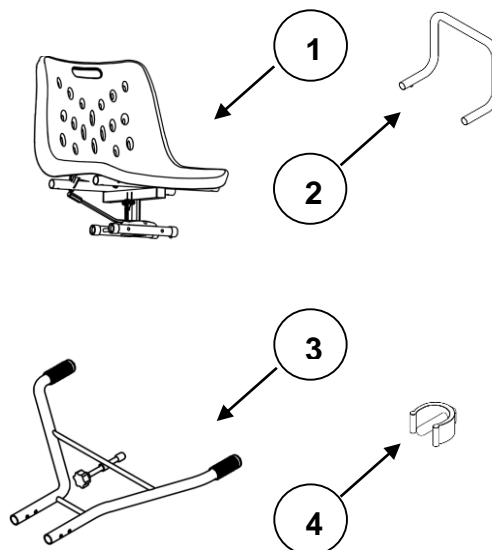


Diagram: 1

NRS Healthcare,
Sherwood House, Cartwright Way, Forest Business Park,
Bardon Hill, Coalville, Leicestershire, LE67 1UB, United Kingdom.

Tel. Head Office: 0330 100 0098
Tel. Customer Services: 0345 121 8111
Email: customerservice@nrshealthcare.co.uk

Assembly and Installation

Remove all parts from the packaging and check they are not damaged. If you see, or suspect a fault, do not use, contact your dealer immediately.

Product Assembly

Slide each armrest (A) into the locating tubes on the seat assembly until the spring button clips “snap” into position, see diagram 2

Slide the mounting frame (B) into the locating tubes on the seat assembly, note the frame is width adjustable, choose the correct width to suit your bath. Please refer to the instruction for correct fitting

Assembling to a bath

Place the seat onto flat bath ledges.

IMPORTANT: At least 50mm (2") of ALL the mounting frame feet (C) must rest on the flat sides of the bath

If necessary extend the mounting frame by removing the fixing e-clips (D), adjusting the width of the frame so that it is closest to the external width of the bath and replacing the clips, check they are all replaced and secured correctly.

Secure the mounting frame using the clamps (E) under the seat by turning the hand wheels until secure against the walls of the bath on both sides. Check the frame is secure, but do not over tighten as this could damage the bath. The bath seat is now ready for use


Safety Note: If the Seat is removed and replaced over the bath, it is important the fixing clamps are re-adjusted and tightened as detailed above

Using the Bath Seat

- Raise the locking handle (F) and rotate until the seat is facing the user, lower the handle and lock into position.
- Transfer user into seat. Raise locking handle and rotate slowly into bath, taking care when raising legs over the side of the bath.
- Ensure seat is locked into position and no longer rotates freely before commencing washing.

Care & Maintenance

It is recommended the product is wiped down after use, or at regular intervals to maintain the quality finish – see cleaning instructions

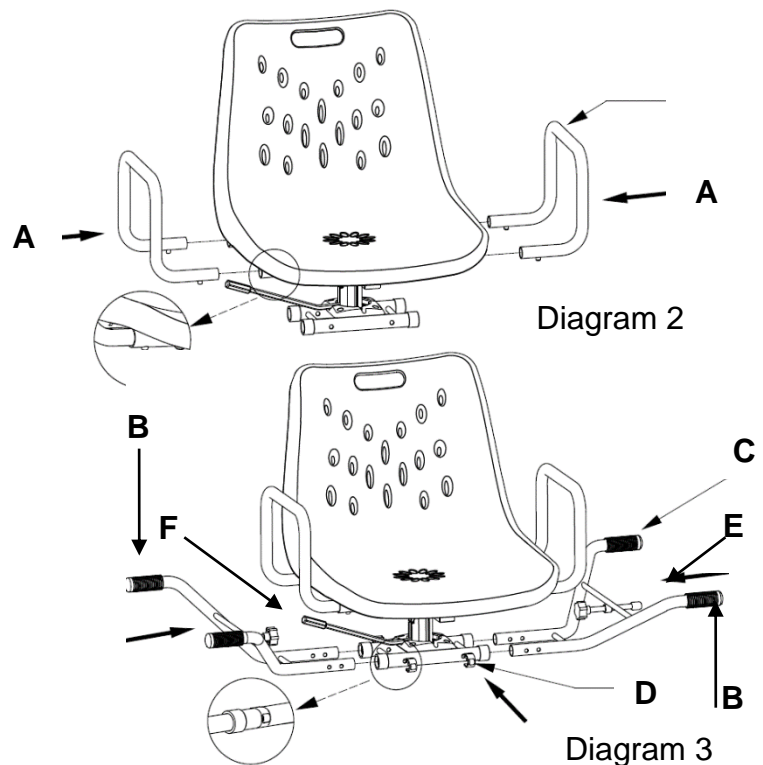

Cleaning Instructions

The swivel Bath seat has a powder coated aluminium frame with a moulded polypropylene seat. Clean all parts regularly, wiping down with a mild detergent /disinfectant solution. Avoid the use of abrasive cloths and cleaners, as they can damage plastic parts. Inspect the product regularly for signs of wear or damage

Maintenance

Inspect the product at regular intervals, pay particular attention to the swivel and lock movement.

Check all fixings, ensure they are secure. Check all parts for signs of wear or damage. If the product is not moving freely or you suspect a fault do not use – contact your dealer immediately

Product Guarantee

NRS Healthcare, guarantee this product for a period of 12 months from the date of purchase against faulty materials and workmanship. This guarantee does not cover, abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights

This product is part of a range of over 3000 daily living aids available from NRS Healthcare.

To view the full range, please visit our website, or request a catalogue by calling NRS Healthcare.