

F25145

Novelle[®] Clip-On Raised Toilet Seat

⚠️ General Safety Warning ⚠️

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

Maximum User Weight



1. INTENDED USE

The Novelle Clip On Raised Toilet Seat will provide a 3" (75mm) raise to an existing toilet seat, providing an overall raise of 4" (100mm). It provides a secure fix without the need for brackets. It can be fitted to the majority of standard toilet seats



3. ASSEMBLY/INSTALLATION

Ensure the existing toilet seat assembly is securely fitted. Lift the toilet seat lid.

3.1 FITTING INSTRUCTIONS:



Squeeze the rear open "horseshoe" sections together



Fit the front of the Novelle Seat into the toilet seat.

2. SAFETY

Warning: The maximum user weight must not be more than 190Kg (30st).

Always ensure that the existing toilet seat is securely fitted prior to installing the Novelle.

Always check the Novelle is clean and undamaged and all the buffers are in place.



Push rear sections on to the seat releasing pressure on the sides, once in position.

Note: Check the clips have engaged correctly by lifting. If correctly fitted, the existing seat will lift with the Novelle.

4. CLEANING INSTRUCTIONS.

Clean regularly, wipe down using a mild detergent or disinfectant solution.

Do not use an abrasive detergent, cleaner or cloth.

This product can be autoclaved up to 800C, and can be immersed in hot dilute solutions of bleach or antiseptic (not exceeding 800C)

Allow product to fully dry prior to use/storage.

5. PRODUCT DISPOSAL

This product should be thoroughly cleaned prior to disposal.

This product is manufactured from plastic and can be disposed of through Local

Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

6. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 12 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover; abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS HEALTHCARE Sherwood House, Cartwright Way, Forest Business Park
Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services: 0345 121 8111

Email: customerservice@nrshealthcare.co.uk

NRS Healthcare is a leading supplier of community equipment and daily living aids.
Please visit one of our websites to view our full range of products:
www.healthcarepro.co.uk or www.completecareshop.co.uk