

# USER INSTRUCTIONS

## HOME HELPER TROLLEY

M03792

Home Helper Trolley Brown

### ⚠ GENERAL SAFETY WARNING ⚠

Please make sure that you read and follow these instructions carefully. Failure to do so can result in injury. Keep these instructions in a safe place for future reference.

### INTENDED USE

The Home Helper Trolley is ideal for indoor home use. The strong and versatile frame is height adjustable and is fitted with two trays to provide a safer means for users to move items around the home. The lower tray is set forward to allow users to walk naturally when pushing.




### Contents:

1. Trolley Frame x 1
2. E-Clips x 4
3. Legs, Castors & E-Clips (pre-assembled) x 4
4. Handles x 2

### ASSEMBLY

#### 1. Attach the handles to the Trolley Frame:

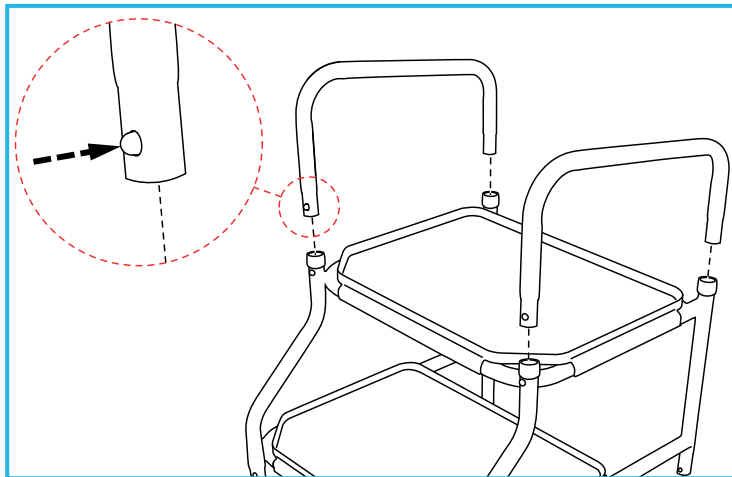

### SAFETY

- Do not use the trolley as a walking aid.
- Do not use the trolley to assist when attempting to stand from a seated position.
- Do not attempt to sit on the trolley.
- Do not overload the trolley trays. When possible it is advisable to position items centrally.
- Do not use the trolley outdoors.

### UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

Slot the handles into the top of the trolley frame.


**Note:** Make sure the raised end faces towards the front of the trolley.

Push and hold the handle button-clips, then push the handles into the trolley frame until the button-clips click into position in the holes.

**Note:** Make sure that the button clips have fully engaged the holes.


## 2. Attach the legs to the Trolley Frame:

The castors are supplied pre-assembled inside the trolley legs.

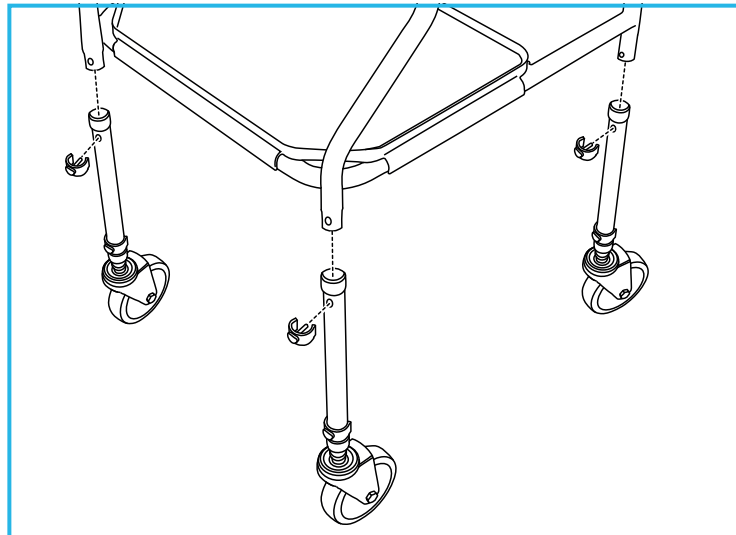

Slot the leg tube over the ends of the trolley frame. Align the holes and insert an E-clip. Repeat for all 4 legs.

## 3. Adjusting the Height:

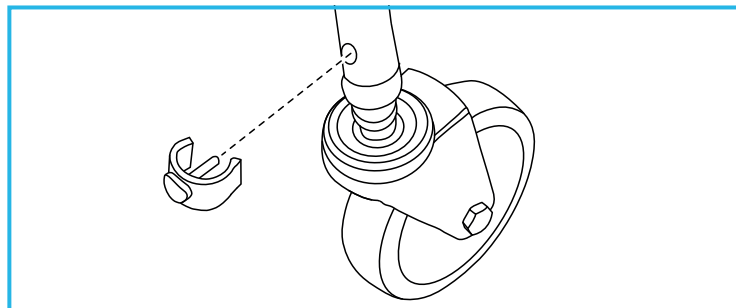

Remove the E-clip securing the castor in position. Slide to the required height. Align the holes and re-insert the E-clip. Repeat for all 4 castors.

**Note:** Make sure all 4 castors are set to the same height to prevent instability.

## CLEANING

Clean regularly using a general purpose, neutral detergent and warm water. If required the trays can be removed for cleaning, simply unclip from the frame.

Please check the suitability of the cleaning product before use. Do not use abrasive cleaners/cloths as this will damage the surface.

Allow product to dry thoroughly before use.

## MAINTENANCE

Regularly check the condition of the castors and remove any build-up of dirt or debris.

Check that all the fixing clips are inserted correctly and that there is no excessive movement.

Periodically check the trolley for signs of wear. Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

## PRODUCT DISPOSAL

This product should be thoroughly cleaned prior to disposal.

This product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

## PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 12 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover; abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.



**NRS HEALTHCARE** Sherwood House,  
Cartwright Way, Forest Business Park, Bardon  
Hill, Coalville, Leicestershire, LE67 1UB, UK.



**NRS HEALTHCARE** 4C Altona Road, Blaris  
Industrial Estate, Lisburn, Co Antrim,  
Northern Ireland BT27 5QB

Tel: Customer Services: 0345 121 8111

Email: [customerservice@nrshealthcare.co.uk](mailto:customerservice@nrshealthcare.co.uk)

**NRS Healthcare** is a leading supplier of community equipment and daily living aids.

Please visit one of our websites to view our full range of products:

[www.healthcarepro.co.uk](http://www.healthcarepro.co.uk) or [www.completecareshop.co.uk](http://www.completecareshop.co.uk)