

User Instructions

M11077 Nuvo Adjustable Bath Step

General Safety Warning

Please ensure these instructions are fully read and implemented. Failure to do so may result in injury. Retain these instructions in a safe place for future reference.

1.0 Intended Use

The Nuvo Adjustable Bath Step is designed to assist users transferring into and out of the bath by reducing the step-over height. It can also provide assistance when using the toilet or sink in a bathroom environment.

2.0 Safety Precautions

Do not exceed maximum user weight 317kg, 50st.

The maximum step height is 175mm (7") using 6 step sections (2 packs).

Do not stack more than 6 step sections together.

Always ensure the bottom section of the step has 4 support pillars inserted in the base.

3.0 Unpacking

Remove packaging and check all parts carefully. If any parts appear damaged do not assemble and contact your supplier immediately.

M11077 Contents: Step 50mm (2") X 3
Support Pillar X 4

4.0 Assembly

4.1 For a single step:

50mm (2") Step Height:

- Use a single step section.
- Turn the section upside down and lay flat.
- Push the four support pillars into position, figure 2.
- Turn the step over to stand on the pillars and place in the desired location.

75mm (3") Step Height:

- Place a second step section on top of the first.
- Push down firmly to secure, figure 3.

100mm (4") Step Height:

- Place a third step section on top of the previous two.
- Push down firmly to secure, figure 4.

Maximum User Weight



Figure 1 – M11077

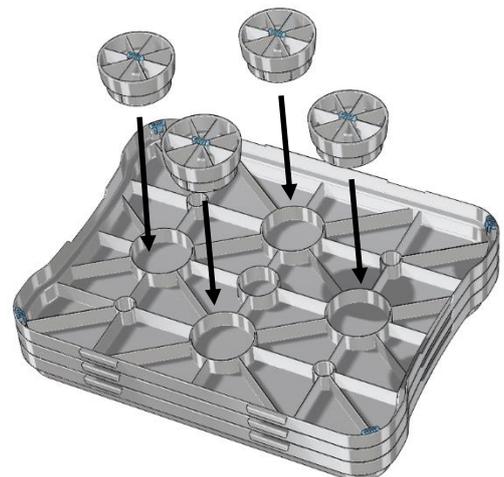


Figure 2 – Fitting the Support Pillars

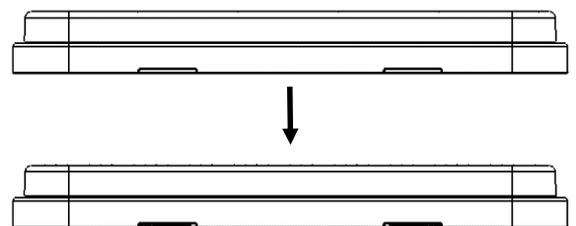


Figure 3 – Adding Step Sections



Figure 4 – 100mm (4") Step Height

4.2 For A Step Sequence:

This set-up provides a series of 2 steps, the first at 50mm (2"), the second at 75mm (3"), figure 5.



Figure 5 – Step Sequence

Please Note: An additional set of 4 support pillars is required to achieve this set up. See Section 7.0 for details.

- Turn two of the three steps upside down and lay flat. These will become the base steps.
- Push the support pillars into position, four on each base step.
- Turn the base steps over to stand on the pillars.
- Link the two sections together by overlapping the cut out section on one with the protruding lip on the other, figure 6.
- Push down firmly to secure.
- Place the third step section on top of one of the base sections.
- Push down firmly to secure.

To achieve a similar set up with greater height or more steps in the sequence, additional packs of steps and support pillars must be purchased.

Please Note: The safe maximum step height is 175mm (7"). This uses 2 packs, 6 step sections. Do not stack more than 6 sections together.

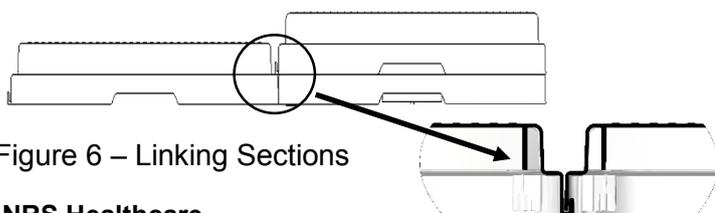


Figure 6 – Linking Sections

NRS Healthcare

Sherwood House, Cartwright Way,
Forest Business park, Bardon Hill,
Coalville, Leicestershire, LE67 1UB, UK

5.0 Cleaning

Clean regularly using a general purpose, neutral detergent and warm water.

Please check the suitability of the cleaning products before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.

To disinfect, wipe with a dilute bleach solution and rinse thoroughly with fresh water.

This product can be autoclaved on cycles up to 80°C.

Allow the product to dry fully prior to use/storage.

6.0 Maintenance

Check that all support pillars are inserted correctly and any additional sections are securely fitted.

Inspect the product regularly for signs of wear or damage. Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

7.0 Spares & Accessories

M24345 – Support Pillars – Set of 4

For more information or to purchase please visit our website www.nrshealthcare.co.uk or call Customer Services 0345 121 8111.

8.0 Product Disposal

Product is manufactured from plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through local Trade Waste Collectors.

9.0 Product Guarantee

NRS Healthcare guarantee this product for a period of 12 months, from the date of purchase, against faulty materials and workmanship. This guarantee does not cover; abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

Tel. Head Office: 0330 100 0098

Tel. Customer Services: 0345 121 8111

Email: customerservices@nrshealthcare.co.uk

NRS Healthcare is a Leading Supplier of Community Equipment and Daily Living Aids, to view our full range, visit our website:

www.nrshealthcare.co.uk