



Quad Canes

Fixing and Maintenance Instructions



VP159AA

Small Base Quad Cane
Height Adjustable 715-965mm



VP159AB

Large Base Quad Cane
Height Adjustable 780-1005mm

Maximum User Weight - 127kg (20st.)

Do not exceed the weight limit stated – doing so could put the user at risk.

THESE ARE NOT MEDICAL DEVICES.

This file is available to view and download as a PDF at www.aidapt.co.uk. Sight impaired customers can use a free PDF Reader (such as adobe.com/reader) to zoom in and increase the text size for improved readability.

READ BEFORE USE

Before using your new product, you and each person that may assist you in its use should take time to read this manual and follow all instructions.

INTRODUCTION



Please examine your cane for any visible damage before using. If you see any damage or suspect a fault, please contact your supplier.

In this case, please do not use your product as it may compromise your safety.

ADJUSTMENT OF LEGS (IF APPLICABLE)

Remove the 'E' clip from the extension leg and extend all legs evenly to the desired height ensuring you do not exceed the last punched hole of the extension leg. (see fig. 1)

Once you have achieved the height you need, re-install the 'E' clip ensuring that it has passed through the extension leg and is fitting neatly and snugly around the commode leg. (see fig. 2)

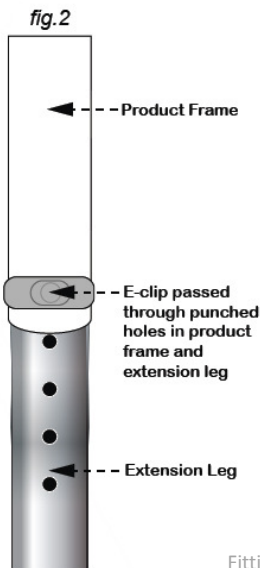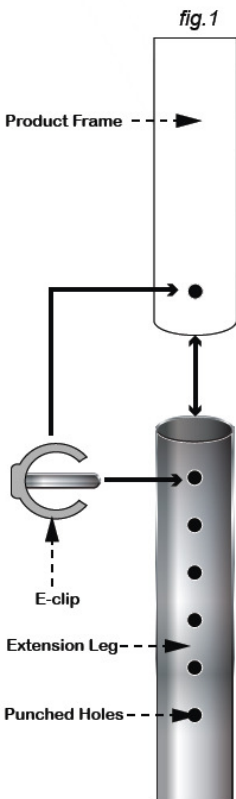

SAFETY

- Avoid wet floors and ice to make sure of good grip on surfaces
- Always ensure E clip is correctly connected/fitted
- Check rubber ferrules for wear
- Do not exceed weight limit

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

This includes:

- Tightness of all nuts/ bolts/castors/rivets
- Other screw in/ bolt in/ push in components.
- Also check all upholstery for security, splits, etc.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

When reissuing, a risk assessment may be required as to the suitability of the chair for a particular user.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

Do not allow children or unauthorised persons to play with the equipment or use without proper supervision.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX
Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.co.uk
Email: accounts@aidapt.co.uk • adaptations@aidapt.co.uk • sales@aidapt.co.uk