

User Instructions

Suspended Bath Seat



Suspended Bath Seat S54547











Read and follow these instructions carefully. Failure to do so can result in injury. Keep these instructions in a safe place for future reference.

Intended Use



To provide a raised bathing seating position in a standard, parallel-sided bath.

Symbols


Symbols used on the product and in these instructions.

	Manufacturer		Batch Reference Code		Consult the instructions for use
	Item/Catalogue number		Important Note		Recycling Symbol
	Maximum Load Weight		Warning		

Safety



Do not exceed the maximum load weight of 136 kg (21 st).



Inspect the product before use. Contact the service provider if you see any damage.

Suitable for most popular parallel-sided baths with flat top edges of equal height. Not suitable for corner baths, or baths with uneven or profiled top edges.

The edges of the bath must be strong enough to support the load and larger than 55 mm wide to support the ends of the bath seat legs.

MAKE SURE:


- The seat is level, correctly positioned so that you can comfortably reach the bath taps and the seat is adjusted to allow safe and comfortable use.

DO NOT:

- Stand on any part of the product.

ALWAYS:

- Sit towards the centre of the seat.

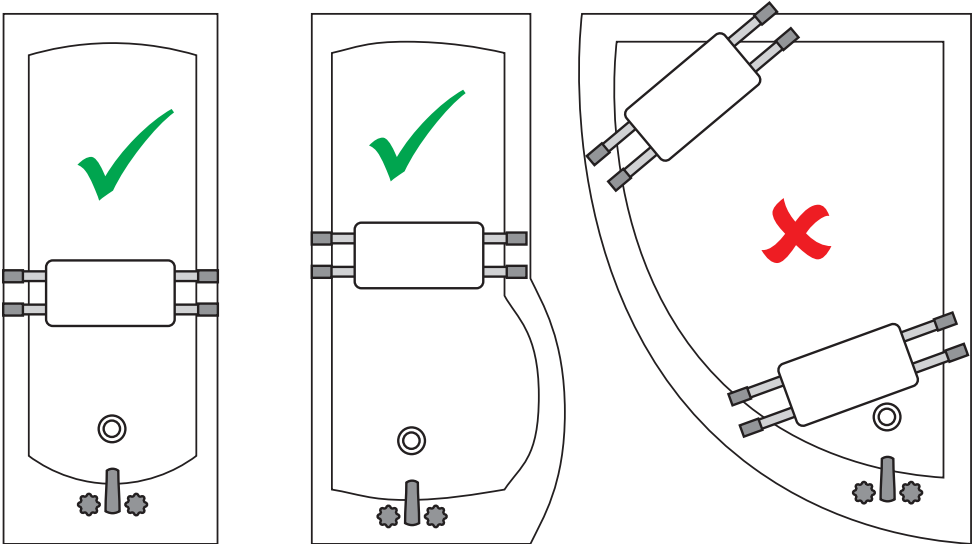


Unpacking

Remove the packaging and check all parts carefully.



Contact your supplier immediately if parts are missing or damaged. Do not try to assemble.


Contents

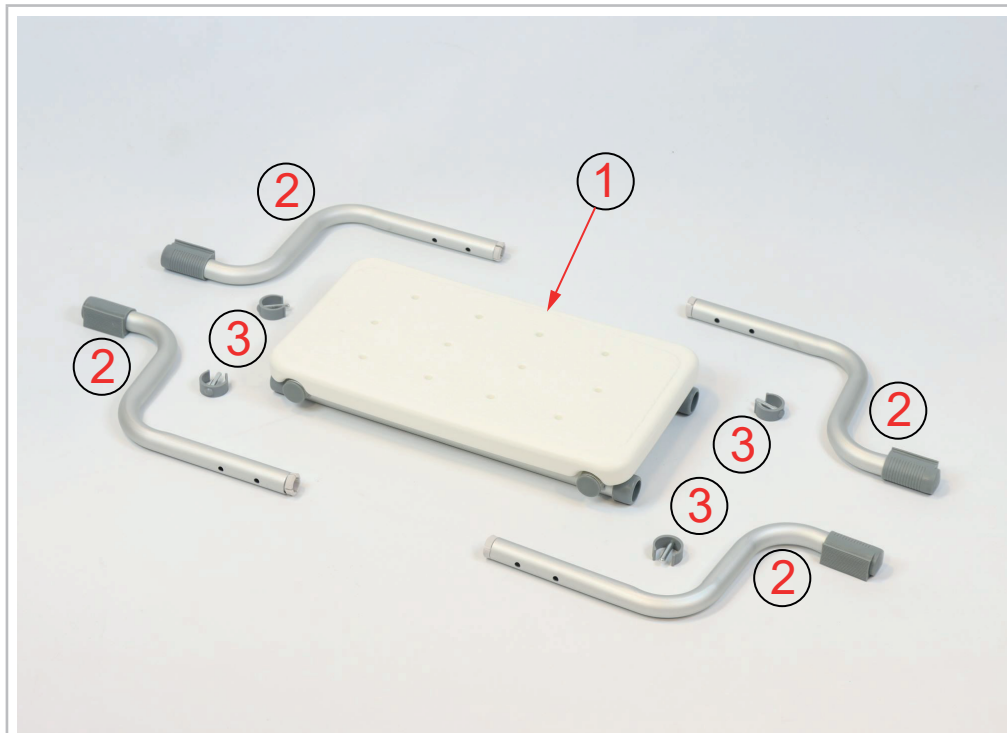

- 1. Seat
- 2. Legs x 4
- 3. E-clips x 4

Assembly/Installation

1

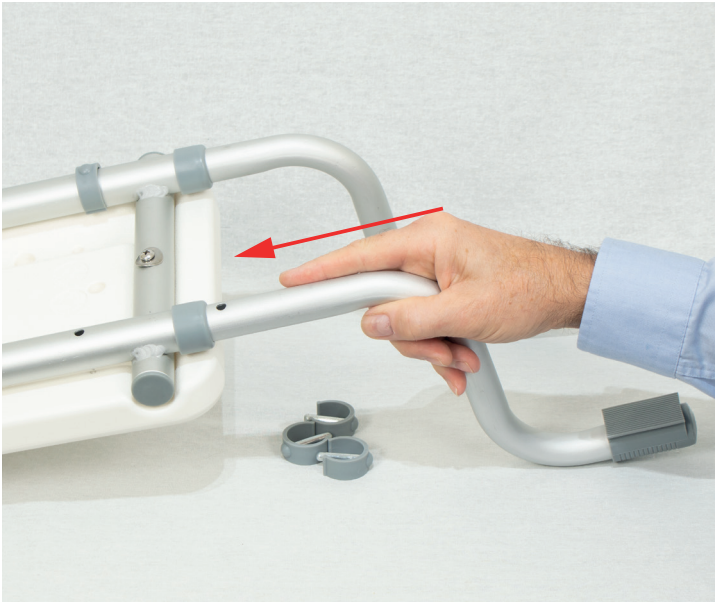

Put the seat upside down on a clean surface.

Insert the four legs into the frame under the seat.

Align the holes and insert the E-clips.



2



Seat assembled.

Adjusting and Using the Seat

1




Check that the seat sits safely across the bath, so that the grey parts of the legs sit fully on the edges of the bath.

If you need to adjust the width of the seat, remove the E-clips and adjust the legs to the next position. See Assembly/Installation.

Fit the seat so that you can reach the bath taps without struggling.



2

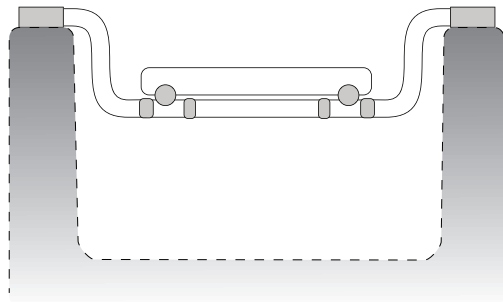

Check you have assembled the seat legs correctly and that the seat sits inside the bath.

Maintenance

Inspect the product at regular intervals and before re-use for signs of general wear or damage.
Report to the supplier any suspected faults or damage and take the product out of use immediately.
Check the fixings at regular intervals.

Cleaning

Clean regularly using a general purpose, neutral detergent and warm water. Check the suitability of the cleaning product before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.

To disinfect, wipe with a dilute bleach solution, and clean with a damp cloth.

Allow product to fully dry before using and storage.

Product Disposal

Thoroughly clean this product before disposal and recycle where possible.

This product is made of metal and plastic. It can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.



Recycle any cardboard packaging.

Product Guarantee

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

If you have any problems with our products, please contact us. See the end of this document for our contact details.

We are a leading supplier of community equipment and daily living aids.
Please visit one of our websites to see our full range of products.

NRS Healthcare. Sherwood House,
Cartwright Way, Forest Business Park,
Bardon Hill, Coalville,
Leicestershire, LE67 1UB, UK

Tel: +44 (0)345 121 8111
Email: customerservice@nrshealthcare.co.uk
Web: www.healthcarepro.co.uk
or www.completecareshop.co.uk