

## User Instructions Long Handled Sponge

**M09873 – Long Handled Sponge – 15” (380mm)**

**M09885 – Long Handled Sponge – 24” (610mm)**

### General safety Warning

Please ensure these instructions are fully read and implemented. Failure to do so may result in injury. Retain these instructions in a safe place for future reference.

### 1.0 Intended Use

The Long Handled Sponge provides additional reach when bathing or showering for those with a reduced shoulder and upper body movement. The wire shaft is flexible and can be bent to a suitable angle as required to improve access based on the client's ability.


### 2.0 Safety

Do not use the sponge if it is damaged or contaminated.

Always rinse sponge after use to prevent build-up of bacteria.

### 3.0 Unpacking

Remove all packaging and check product for signs of wear or damage. Any suspected faults should immediately be reported to supplier and the product left unused.

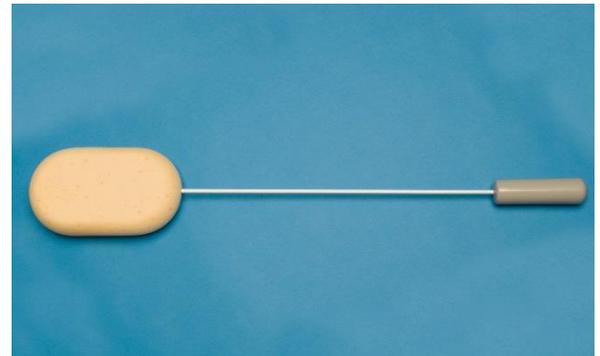

### 5.0 Use

Use in the bath or shower as a normal bathing sponge.

### 4.0 Cleaning

Rinse thoroughly with fresh water after use.

Allow the sponge to air dry after use and cleaning.

Leave in a dry, draughty environment to prevent build-up of damp.

### NRS Healthcare

Sherwood House, Cartwright Way,  
Forest Business park, Bardon Hill,  
Coalville, Leicestershire, LE67 1UB, UK

Tel. Head Office: 0330 100 0098

Tel. Customer Services: 0345 121 8111

Email: [customerservices@nrshealthcare.co.uk](mailto:customerservices@nrshealthcare.co.uk)

**NRS Healthcare** is a Leading Supplier of Community Equipment and Daily Living Aids, to view our full range, visit our website:

[www.nrshealthcare.co.uk](http://www.nrshealthcare.co.uk)