

USER INSTRUCTIONS

NOVELLE[®] CLIP-ON RAISED TOILET SEAT

F25145

Novelle[®] Clip-On Raised Toilet Seat

⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

Maximum User Weight



1. INTENDED USE

To provide a 3" (75 mm) raise to an existing toilet seat, providing an overall raise of 4" (100 mm). It provides a secure fix without the need for brackets. It can be fitted to the majority of standard toilet seats.



2. SAFETY

Warning: The maximum user weight must not be more than 190 kg (30 st).

Always ensure that the existing toilet seat is securely fitted prior to installing the Novelle.

Always check the Novelle is clean and undamaged and all the buffers are in place.

3. ASSEMBLY/INSTALLATION

Ensure the existing toilet seat assembly is securely fitted. Lift the toilet seat lid.

3.1 FITTING INSTRUCTIONS:



Squeeze the rear open "horseshoe" sections together.



Fit the front of the Novelle Seat into the toilet seat.



Push rear sections on to the seat releasing pressure on the sides, once in position.

Note: Check the clips have engaged correctly by lifting. If correctly fitted, the existing seat will lift with the Novelle.

4. CLEANING INSTRUCTIONS.

Clean regularly, wipe down using a mild detergent or disinfectant solution.

Do not use an abrasive detergent, cleaner or cloth.

This product can be autoclaved up to 80C, and can be immersed in hot dilute solutions of bleach or antiseptic (not exceeding 80C).

Allow product to fully dry prior to use/storage.

5. PRODUCT DISPOSAL

This product should be thoroughly cleaned prior to disposal.

This product is manufactured from plastic and can be disposed of through Local Authority Household Waste and Recycling Centres.

or through Local Trade Waste Collectors.

6. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS HEALTHCARE Sherwood House, Cartwright Way, Forest Business Park
Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services: +44 (0)345 121 8111

Email: customerservice@nrshealthcare.co.uk

NRS Healthcare is a leading supplier of community equipment and daily living aids.

Please visit one of our websites to view our full range of products:

www.healthcarepro.co.uk or www.completecareshop.co.uk