



Solutions for Daily Living



Solutions for Daily Living

11724 Willake Street
Santa Fe Springs, CA 90670
(562) 282-0244
Fax: (310) 305-1718

Email: info@columbiamedical.com
Website: www.columbiamedical.com
© 2010 Columbia Medical Mfg., LLC

Stainless Steel Extension/ Rolling Shower Bases

For use with Ultima™ Bath Chair

Model Numbers:

8880 (without casters)

8890 (with casters)



Ultima shown above
with 8880 extension
base.



Ultima shown above with 8890 rolling
base and optional accessories.

COLUMBIA MEDICAL MFG. LLC warrants this product to be free of defects in material and workmanship. Our obligation under this warranty is limited to repair or replacement of any part or entire unit at our option for a period of one year from date of delivery to the original purchaser. The warranty does not include cost of inconvenience, property damage, misuse, abuse, accident or similar incidents. The warranty will not apply if product has been tampered with or repaired by unauthorized individuals.

ATTENTION

Your Ultima™ Base has been built to the highest standards of quality to ensure years of superior service. Please take the time to read this informative instruction guide and pay particular attention to the safety information provided.

Here are some additional precautions you can take to reduce the risk of injury:

1. Read and understand ALL instructions.
2. Determine, with a physician or therapist, the suitability of this product for the intended user.
3. To reduce the risk of injury or drowning, NEVER leave the user unsupervised or unattended while in the bath.
4. SAVE THESE INSTRUCTIONS.

If you require additional assistance, please call Columbia Medical Customer Service at **(800) 454-6612**, or e-mail info@columbiamedical.com

Thank you for choosing Columbia Medical.

Before You Begin

- NEVER position the Ultima legs at a complete vertical position. This will cause instability and can cause severe injury.
- The Ultima Bath Chair, when used with the 8890 Rolling Base, IS NOT INTENDED to be used as a wheelchair.
- Use EXTREME CARE when positioning a user and while moving the unit over obstacles on the floor.
- The 8880 Extension Base and 8890 Rolling Base are not for use with chairs other than the Columbia Ultima Bath Chair.


Assembly Instructions

The stainless steel bases are designed to be used with your Columbia Ultima™ Stainless Steel Bath Chair. Both bases raise the height of your bath chair. Advantages of a higher bath chair include easier transfers and a reduced risk of back strain for the care giver. You will need a hand-held shower hose in your tub for proper bathing.

Step 1:

Remove unit from the box and place on a flat surface. Loosen the TWO blue adjustment knobs by turning them counter-clockwise. See Fig. 1

If using the 8800 extension base (without the casters), make sure to place the base in the tub before moving on to Step 2.

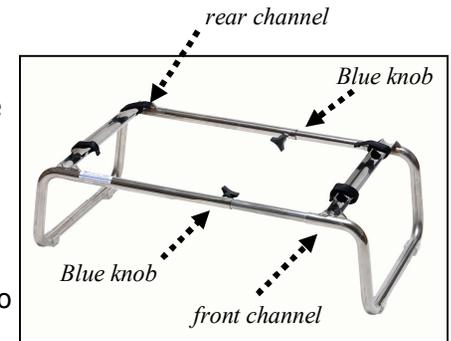

(Fig. 1) 8880 shown in example above

Step 2:

Adjust the legs on your Ultima bath chair to the desired position. Place the rear legs onto the rear channel of the 8880/8890 base unit; the horizontal base of the legs will fit into the support channel. See Fig. 1

Slowly lower the front legs of the Ultima chair into the second channel. If needed, adjust the length of the 8880/8890 base to accommodate the front legs of bath chair. See Fig. 2



(Fig. 2) 8880 shown in example above

Step 3:

Tighten the blue adjustment knobs by turning them clockwise.

Step 4:

Connect the holding straps, pulling tight to make sure the Velcro is properly engaged.