

# USER INSTRUCTIONS

## NRS FOLDAWAY FOOTREST

M99631

NRS Foldaway Footrest

### ⚠ General Safety Warning ⚠

**Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.**

#### 1. INTENDED USE

This height adjustable footrest offers a choice of three different height positions. The highest position elevates the legs to help stimulate circulation and ease tired legs. The footrest can be angled in various positions to rest legs comfortably. The footrest can also be used flattened to a neat footstool for resting thighs and keeping feet warm.

#### 2. SAFETY

Do not use this product until you have read these instructions.

For maximum stability ensure the hinge on the base is facing the chair as shown in Figure 1.


**DO NOT SIT** on the footrest. The product has a maximum weight limit of 18 kg (2.8 st) and is not suitable for sitting on.

#### 3. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

**Note:** The front bar is tucked inside the fabric cover to prevent damage in transportation. Before first use remove the bar from inside the cover and allow to swing freely.

#### 4. ASSEMBLY/INSTALLATION

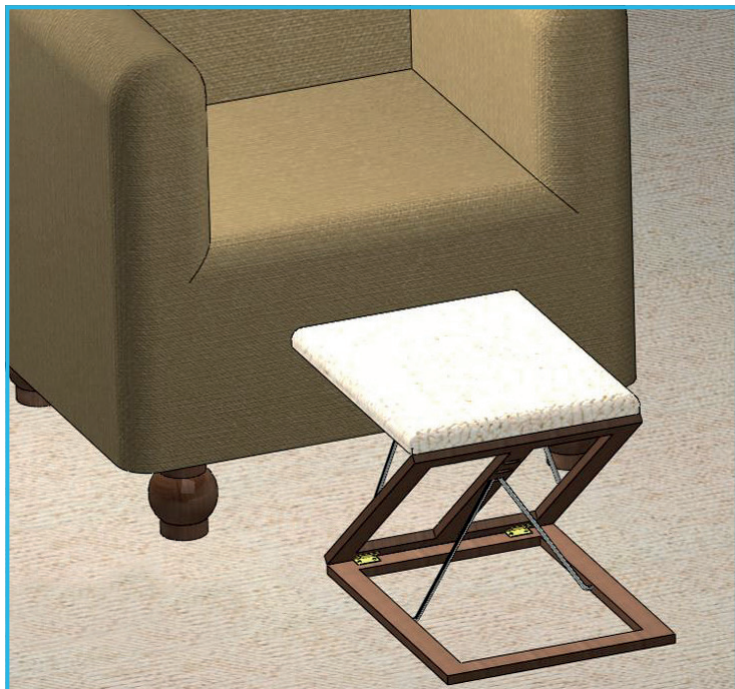

**Figure 1:**

#### Maximum User Weight




1. Position the footrest as shown in Figure 1. The footrest can then be set at the required height and angle.

**Note:** Note the angle can be either forward or backward facing, whichever is the most comfortable.



**Figure 2:**

2. When the footrest is set in the desired position, the front and rear bars must be engaged in the appropriate slots as shown in Figure 3.

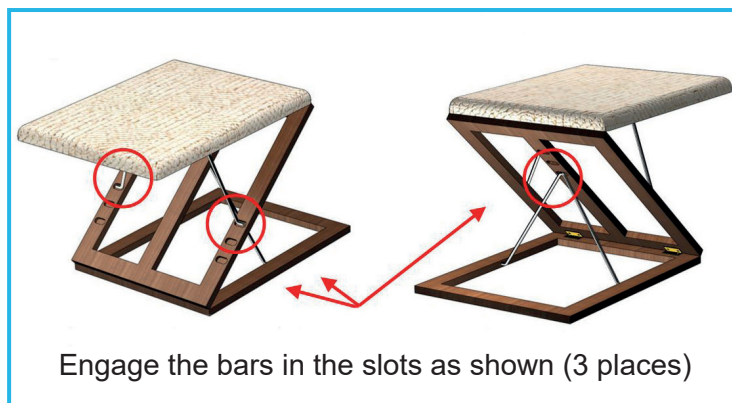

**Figure 3:**

#### 5. MAINTENANCE

Periodically check that the support bars and hinges are in good working order.

#### 6. CLEANING INSTRUCTIONS

The frame assembly should be wiped down with a cloth, dampened with a mild detergent solution.

The top cover can be removed for cleaning - **HAND WASH ONLY**, using a mild detergent.

## 7. PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from wood, metal and fabric and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

## 8. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

**NRS HEALTHCARE** Sherwood House, Cartwright Way, Forest Business Park  
Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services: +44 (0)345 121 8111

Email: [customerservice@nrshealthcare.co.uk](mailto:customerservice@nrshealthcare.co.uk)

**NRS Healthcare** is a leading supplier of community equipment and daily living aids.  
Please visit one of our websites to view our full range of products:  
[www.healthcarepro.co.uk](http://www.healthcarepro.co.uk) or [www.completecareshop.co.uk](http://www.completecareshop.co.uk)