

Introduction

The Days Healthcare 613 Folding bed cradle is manufactured from chrome plated steel tube and is designed to be held between the mattress and the bed base whilst the upper section supports the sheets or blanket covers. When not in use, the bed cradle can be folded flat for easy storage.



Installation Guide

To unfold the bed cradle, rotate both end frames about their pivots on the centre frame. The wider of the two end frames locates between the bed base and the mattress whilst the narrower end frame supports the sheets or blanket covers, holding them away from the patients body.

NOTE! - Do not lean on the bed cradle or apply heavy loads. Maximum load-5kg.

Cleaning

Wipe down your bed cradle with a mild detergent and warm soapy water. Do not immerse the unit in any liquids, steam clean or pressure wash. Do not use bleach based cleaners or harsh abrasives.

Maintenance

Periodically check the pivot points of the upper and lower end frames to ensure that they are securely in position and rotate freely.

Warranty Information

The Days Healthcare 613 bed cradle is warranted for one year from the date of purchase against faulty workmanship or materials. Please contact your supplier/dealer should a fault occur.

The warranty does not extend to the consequential costs resulting from fault clearance, in particular freight and travel costs, loss of earnings, expenses, etc.

The manufacturer will not accept responsibility for any damage or injury caused by misuse or non-observance of the instructions set out above.