



**THE LOCOMOTOR RANGE**

# **LOCOMOTOR DUO TRANSFER SLIDE**

## **INSTRUCTIONS FOR USE**

To be left with the user.



**LOCO-401B LOCOMOTOR DUO TRANSFER SLIDE**  
**LOCO-400B LOCOMOTOR DUO TRANSFER SLIDE DELUXE**  
**LOCO-402 SPARE SET OF HANDLES**

Registered Design # 3019601

Registered Design # 3019601

### **LOCOMOTOR DUO TRANSFER BOARDS**

The boards are designed to allow non or semi weight-bearing patients to transfer independently or to be assisted by trained members of staff from one seated position to another with or without additional equipment. The LOCOMOTOR DUO TRANSFER BOARDS provide a smooth low friction surface that bridges the gap between two surfaces. For example bed or chair to wheel chair

### **IMPORTANT:**

ALL PATIENT HANDLING TASKS SHOULD BE CARRIED OUT IN ACCORDANCE WITH APPROVED MANUAL HANDLING TECHNIQUES AND AFTER A FULL RISK ASSESSMENT.

### **USE of the LOCOMOTOR DUO TRANSFER BOARDS**



Before every use, check that the board is clean and free from cracks or other damage.  
Always keep your LOCOMOTOR DUO TRANSFER BOARD clean.

### **POSITIONING:**

- ALWAYS ENSURE THAT THE NON-SLIP STRIPS ARE FACING DOWNWARDS AND ARE IN MAXIMUM CONTACT WITH THE SURFACES BETWEEN WHICH THE PATIENT IS TRANSFERRING AND THAT THESE SURFACES ARE AS CLOSE TOGETHER AS POSSIBLE.
- ENSURE SURFACES HAVE NO RISK OF MOVING DURING THE TRANSFER AND ARE POSITIONED TO FACILITATE A 'SIDE TRANSFER'.
- ORIENTATE THE BOARD WITH ITS STRAIGHT SIDE TOWARDS THE PATIENT
- THE TRANSFER SHOULD BE MADE WITH THE PATIENT MOVING 'SIDE WAYS' IN A SEATED POSITION SLIDING ALONG THE LENGTH OF THE BOARD.

### **METHOD OF USE**

Commonly the LOCOMOTOR DUO TRANSFER BOARD is positioned by sliding it beneath one hip whilst the patient is in a seated position. Then either independently or with assistance from trained staff the patient slides the length of the LOCOMOTOR DUO TRANSFER BOARD to the required seated position. The LOCOMOTOR DUO TRANSFER BOARD is then removed by tilting the patient and sliding it from beneath the patient.

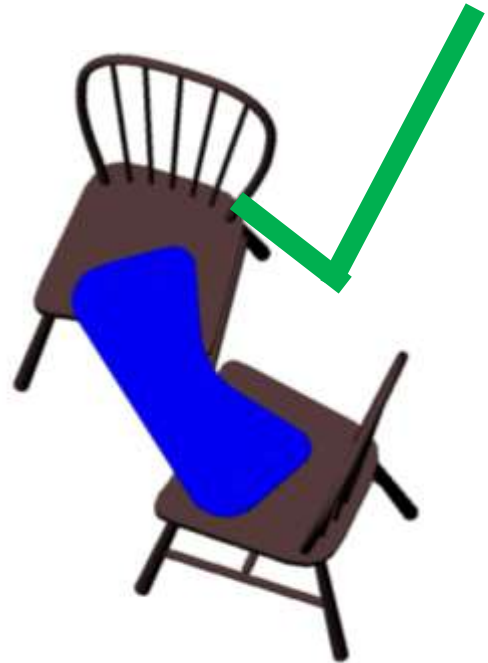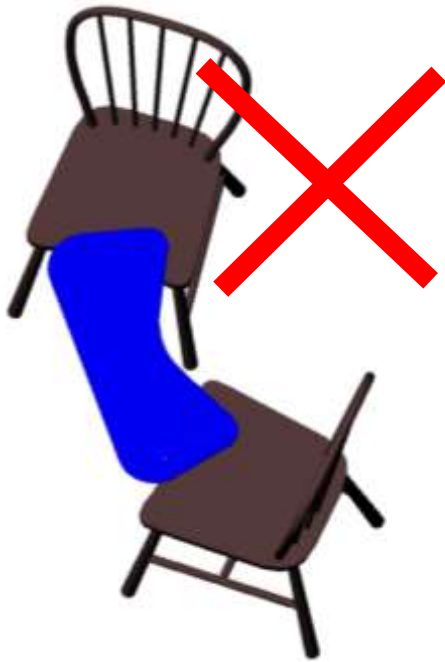

## SPECIFICATION

Maximum patient weight 150 Kg.

Dimensions 75cm long, 22cm wide.

Weight 1420 gm.

LOCO-401B has no handles

LOCO-400B has handles to assist the patients movement

Warranted to specification for 1 year.

Do not store in direct sunlight.

Wash with a proprietary detergent.

Disinfect using a proprietary disinfectant.

Do not use abrasive cleaning materials or substances.

Can be pressure washed at 79C.

## SPARES

LOCO-402 SPARE SET OF HANDLES

## FOR SALES OR TECHNICAL HELP CONTACT-



Select... Health Care (U.K) Ltd.

Unit 2, Albert Rd

Rushden

Northants

NN10 0BU

Tel: (01933) 411729

Fax: (01933) 410213

mail: [info@selecthealthcare.co.uk](mailto:info@selecthealthcare.co.uk)

web Site: [www.selecthealthcare.co.uk](http://www.selecthealthcare.co.uk)

