

USER INSTRUCTIONS

MANOY SLOPED PLATE

F25637

Manoy Sloped Plate - Small

F25650

Manoy Sloped Plate - Large

⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

1. INTENDED USE

Suitable for one-handed eating, with a high back and low front, the Manoy Sloped Plate lets food slide towards the front, making it much easier to see and scoop up.

Made from lightweight and durable white melamine, this cleverly designed plate is also available in a small size so that meals and snacks are made much more enjoyable.



2. SAFETY

This product is not suitable for microwaves.

3. IN USE



Cutaway Showing Plate Contour

The gentle slope and lipped inner wall provides an inclusive option for those who eat using one hand.

On the underside there is a discreet dimpled pattern to indicate the location of the curved lipped wall for those with visual impairments.

4. CLEANING INSTRUCTIONS.

Clean regularly using a general purpose, neutral detergent and warm water.

This product can be cleaned in a dishwasher.

Allow product to fully dry prior to use/storage.

5. PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from melamine and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

6. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 12 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover; abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS HEALTHCARE Sherwood House, Cartwright Way, Forest Business Park
Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services: 0345 121 8111

Email: customerservice@nrshealthcare.co.uk

NRS Healthcare is a leading supplier of community equipment and daily living aids.

Please visit one of our websites to view our full range of products:

www.healthcarepro.co.uk or www.completecareshop.co.uk