


User Instructions

Standard Folding Support Rail Range

M11247 Long Rail - 760mm (30")

M19945 Short Rail - 550mm (21½")

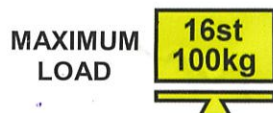

General Safety Warning

This product should only be fitted by a suitably qualified person.

Please ensure these instructions are fully read and understood before installation/ use.

Retain these instructions in a safe place for future reference.



1.0 Intended Use

The Standard Folding Support Rail is a high quality wall mounted steel rail which provides a stable support to assist the user raise and lower themselves for toileting. The rail can be raised back to the wall when not in use or when transferring from the side.

2.0 Safety Precautions

- Read these instructions before installation and use.
- This product must be correctly assembled, installed, and maintained.
- This product must be installed by a suitably qualified, competent, person.
- Never drill through pre-stressed concrete to install a folding support rail.
- Always check for concealed electrical cables and water pipes when installing; also check your legal requirements relating to earth bonding of extraneous metal parts.
- Do not exceed the maximum load - Note this is based on the product being fitted to a suitable surface using appropriate fixings
- Do not install in a shower

3.0 Unpacking

Check product carefully before installation. If you see, or suspect a fault, do not install, contact your supplier immediately

4.0 Installation Instructions

4.1 General

- Only install the product in an area where the mounting surface and substrate are structurally sound - Strengthening modifications to the fixing surface may be required.
- Also check the type of substrate you intend to fix it to e.g. brick/block timber or concrete. Always check that the type of fixing you are going to use is compatible with the substrate.
- **Note: As the substrate can vary considerably, we cannot recommend any particular fixings; if in any doubt; consult a reputable fixings supplier for advice.**
- Ensure that when correctly located, the rail can be raised and lowered without interference from other fixtures and furniture and that transfer can be carried out safely.

4.2 Installation

1 - Position the wall plate against the wall at the height required, check the arm can be raised and lowered without interference.

2 - Mark the position of the holes.

3 - Drill the holes, as appropriate for the fixings/ screws being used. See section 4.1

Installation Continued

4 - Place Wall plate against wall and secure using the chosen fixings. When fitted, check the rail is securely fixed to the wall and that the arm can be raised and lowered without interference. Also check that the friction lock is effective and that the support rail will hold itself at any angle during movement (i.e. it does not drop down on its own).

5 - Important; after installing the product, perform a final tighten and inspection of all fasteners

6 - Test the installation, ensure the rail is rigid, stable and fit for purpose

5.0 Cleaning and Maintenance

5.1 Cleaning

Clean regularly, wiping down using a soft cloth and a mild detergent /disinfectant solution.

Do not use abrasive cleaners.

Do not autoclave.

Always ensure that you wipe dry all equipment and regularly treat joints with a water dispersing agent e.g. WD40.

5.2 Maintenance

Periodically check the stability and rigidity of the product and, where applicable, that screws are tightened and in good condition.

If you see any damage or suspect a fault, do not use and contact your supplier.

NRS Healthcare

Sherwood House, Cartwright Way,
Forest Business Park, Bardon Hill,
Coalville, Leicestershire.
LE67 1UB UK

Tel. Main Office: 0330 100 00 98

Tel. Customer Services: 0345 121 8111

Email: customerservice@nrshealthcare.co.uk



NRS Healthcare is a Leading Supplier of Community Equipment and Daily Living Aids, to view our full range, visit our website:
www.nrshealthcare.co.uk