



PARNELL PREMIER BED RAIL
BRITISH STANDARD BS EN 1970:2000
REGISTERED DESIGN

MAXIMUM USER WEIGHT 112kg / 18st

ASSEMBLE THE BED RAIL AS SHOWN IN THE DIAGRAM • ADJUST THE HANDWHEEL AND POSITION THE TWO FORKS UNDER THE BED FRAME, WITH THE BED RAIL AT A POSITION TO SUIT THE OCCUPANT • TURN THE HANDWHEEL CLOCKWISE UNTIL RESISTANCE IS FELT • CONTINUE TURNING UNTIL SOME OF THE WEIGHT OF THE BED IS BORNE ON THE BED RAIL • THE BED RAIL IS NOW READY FOR USE

- TO REMOVE THE BED RAIL REVERSE THE ABOVE PROCEDURE



THE BED RAIL SHOULD BE PERIODICALLY CLEANED WITH NORMAL DOMESTIC CLEANING MATERIALS, AVOID ACID, ALKALINE OR OTHER CLEANING MATERIALS IN STRONG OR CONCENTRATED FORM AS THEY MAY DAMAGE OR DISCOLOUR THE PRODUCT.

IT IS RECOMMENDED THE TWO HALVES OF THE BED RAIL ARE SEPARATED FOR CARRYING.

PARNELL PRODUCTS LTD.

Tel: 0191 237 2096

Email:

enquiries@parnellproducts.com

www.parnellproducts.com