

M11090

Height Adjustable Toilet Frame with Seat

### ⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

#### 1. INTENDED USE

To assist users when lowering onto and rising from the toilet. Primarily for use over the toilet using the splash guard provided but also for use as a commode with the addition of a potty. The seat lid adds discreet style and helps to improve cleanliness and hygiene.



Figure 1:

#### 2. SAFETY

**WARNING:** The maximum user weight must not be more than 190 kg (30 st).

**DO NOT** attempt to stand or climb on the product.

**MAKE SURE** all legs are set at equal height to avoid rocking.

#### 3. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

#### 4. INSTALLATION INSTRUCTIONS

Product arrives fully assembled.

##### To adjust the height:

- Remove the 'E-clip' and slide the legs to the desired position.
- Align the holes and re-insert the 'E-clip'.
- Repeat for all four legs.

Ensure that all four legs are positioned at the same height to prevent rocking and instability.

#### Maximum User Weight



Place the frame over the toilet ensuring that the internal splash guard is positioned centrally over the toilet bowl.

##### To fit M20303 - Commode Pan



Figure 2: M20303 Commode Pan

Lift the seat and lid to slot the commode pan into position on the frame, see figure 3.



Figure 3: M11090 with Commode Pan (M20303) fitted

#### 5. OPTIONAL FLOOR FIXING KIT

The floor fixing kit provides a permanent fixing solution for the Toilet Frame. It helps to prevent the frame from moving out of position or becoming unstable during use.

## F11572 – Floor Fixing Kit



Available as an accessory from NRS Healthcare. See the end of this document for contact details.

### 5.1 Fitting Instructions

Adjust the toilet frame to the desired width, see Section 4.

#### To fit the legs:

Remove the original leg extensions by removing the 'E-clips' and sliding them out of the frame.

Insert the four leg extensions and position at the desired height.


Align the holes and insert the 'E-clip'.

Ensure that all four legs are secured at the same height to prevent rocking and instability.

Please note the height cannot be adjusted once the legs are secured to the floor.

#### To fix to the floor:

Choose a fixing method which is suitable for the construction of the floor. Use 3 screws per each of the four legs. See figure 4.

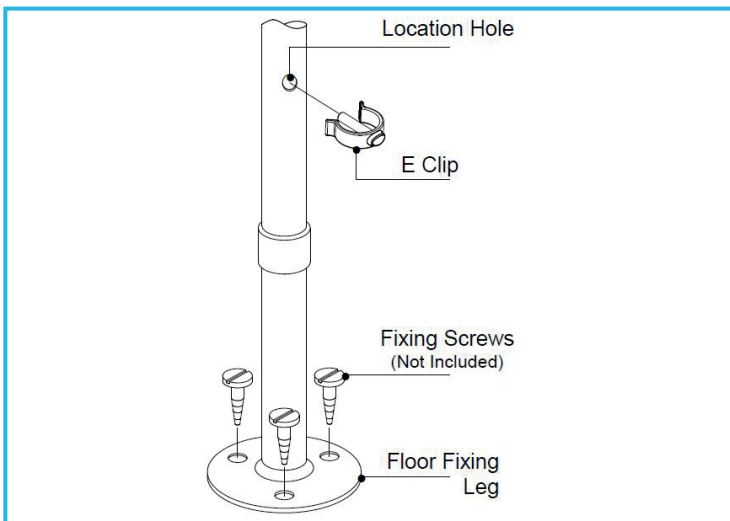

**Figure 4:**

## 6. MAINTENANCE

Check that all fixing clips are inserted correctly and that there is no excessive movement.

If a floor fixing kit has been fitted, check the fixing screws are secure.

Inspect the product regularly for signs of wear or damage, pay particular attention to the condition of the ferrules.

Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

## 7. CLEANING INSTRUCTIONS

Clean regularly using a general purpose, neutral detergent and warm water. Pay particular attention to the toilet seat hinges and underside of lid.

Please check the suitability of the cleaning products before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.

To disinfect, wipe with a dilute bleach solution and rinse thoroughly with fresh water.

Allow the product to dry fully prior to use/storage.

## 8. PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from metal & plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

## 9. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

**NRS HEALTHCARE** Sherwood House, Cartwright Way, Forest Business Park  
Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services: +44 (0)345 121 8111

Email: [customerservice@nrshealthcare.co.uk](mailto:customerservice@nrshealthcare.co.uk)

**NRS Healthcare** is a leading supplier of community equipment and daily living aids.  
Please visit one of our websites to view our full range of products:  
[www.healthcarepro.co.uk](http://www.healthcarepro.co.uk) or [www.completecareshop.co.uk](http://www.completecareshop.co.uk)