

User Instructions M99138

NRS Shower Stool with Swivel Seat

Maximum User Weight



General Safety Warning

Please ensure these instructions are fully read and implemented. Failure to do so may result in injury to the user. Retain in a safe place for future reference.

Intended Use

This sturdy height adjustable stool is for use in a shower. It has a swivelling seat to enable the user to sit and easily position themselves. The seat can be easily removed for cleaning.

Unpacking

This product is delivered flat packed. Remove all parts from the packaging, check for damage or faults, if you see any do not assemble the product contact your supplier immediately.

Contents

| Description | Qty. |
|---------------|------|
| Seat | 1 |
| Leg | 4 |
| Leg Extension | 4 |

Assembly Instructions:

- With the seat upturned push the legs into the seat orifices, as shown in diagram 1 making sure the button clips, snap into position.
- Check the legs are rigid and do not move.
- Fit the Leg Extensions into the legs, again make sure the button clips snap into position.
- Adjust the seat to the desired height by sliding the leg extensions, make sure all the extensions are at the same height

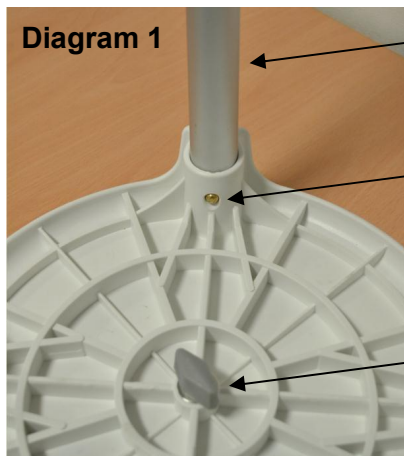


Diagram 1

Leg

Snap Button Clip

Plastic Wing Nut



Important safety information:

- Floor surfaces must be level and even to avoid rocking.
- All four ferrules must be in contact with the floor at all times.
- Periodically check all legs for total movement.
- Ensure the seat is securely fixed
- Do not exceed the maximum user weight limit.
- Inspect at regular intervals for wear and damage.

Cleaning Instructions

Clean regularly, wipe with a soft cloth and a mild detergent/disinfectant solution. Do Not use abrasive cleaners

Note: The swivelling part of the seat can be removed for cleaning by removing the plastic wing nut, as shown in diagram 1

Manufactured for **NRS Healthcare**
Sherwood House, Cartwright Way
Forest Business Park, Bardon Hill
Coalville, Leicestershire. LE67 1UB UK
Tel: 0845 120 4522
Int: +44 (0) 1530 418378
Fax: 0845 121 8112
Email: customerservice@nrs-uk.co.uk

This product is from a range of over 3000 daily living aids available from NRS Healthcare. To view the full range visit www.nrs-uk.co.uk or request a catalogue from the above number.