
# Etac Clean


## Manual

etac®


78644G 20-11-11

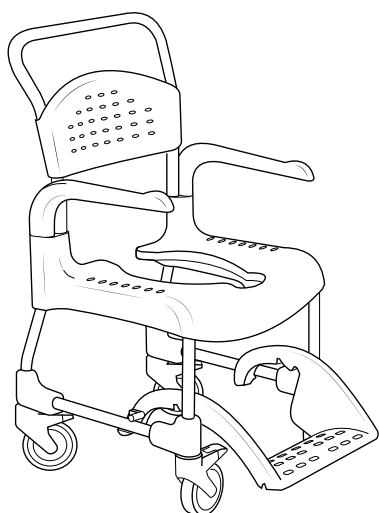Clean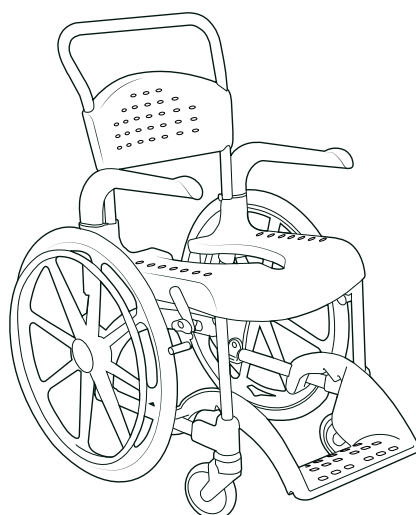

Clean 24"

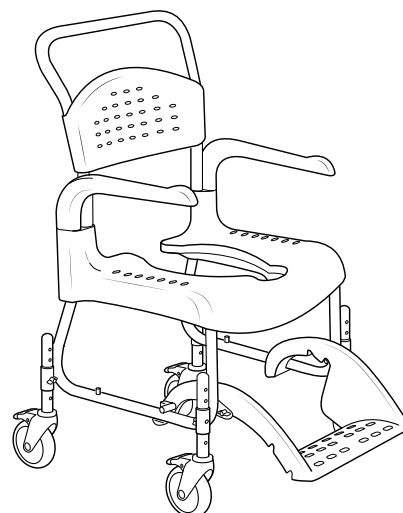Clean HRB

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Clean  
Clean 24"  
Clean HRB

## General

Thank you for choosing a shower chair from Etac.  
In order to avoid injury during handling and use, please read through this manual first.

In the manual, the user is the person sitting in the chair. The assistant is the person operating the chair.

### Intended use:

The Clean family is a range of mobile shower and toilet chairs intended for use indoors when carrying out hygiene care for people with decreased mobility. The shower chair can be used in showers, at sinks or over the toilet, as well as for transfers to and from the hygiene room.

### Contra indications:

Clinical experience shows that there are no harmful effects of using Clean.

Clean has been tested and complies with the requirements stated in EN ISO 10993-1 and SS EN 12182.

This product should be recycled as per national regulations.

This symbol appears in the manual along with text. It indicates instances where the safety of the user or the assistant may be at risk.

The images which are referred to on each page are found at the start of the manual (pages 3 to 7). See Figures A to J for the corresponding text.

Item list: ● = Applies to, ○ = Accessories

See the item list on the right which displays the information applicable to each product.

### This manual covers the following products:

Clean	●	
Clean 24"	●	
Clean HRB (Height adjustable)		●

## Product description & technical data

See Figure A

**A1.** Backrest **A2.** Armrest **A3.** Seat **A4.** Foot support **A5.** Castor wheel **A6.** Product label (warnings, CE-marking) **A7.** Item no label (serial no, production date)

**A8.** Rear wheel **A9.** User brake

**A10.** Height adjustment

● ● ●  
○ ●  
●

## Warranty / maintenance

See Figure B

5-year guarantee against material and manufacturing defects. For terms and conditions, see www.etac.com

### Product lifetime:

Service life 7 years. For complete information regarding the service life of the product, see www.etac.com.

### Maintenance:

To keep the chair in good and safe working order, you should regularly:

- Clean the chair with ordinary non-abrasive detergent. Rinse and dry.
- Keep the castors free of hair and soap deposits, etc.
- If your shower and toilet chair is not in perfect working order, contact your dealer.

### Symbols

**B1.** Clean the chair with ordinary non-abrasive detergent. Rinse and dry.

**B2.** The product can be cleaned at 85 ° for 3 minutes.

● ● ●

## Safety

See Figure B

- Defective chairs must not be used.

### Symbols

**B3.** Risk of pinching at armrest.

**B4.** Do not stand on the footrest, risk of tipping!

**B5.** Maximum user weight: 130 kg.

**B6.** Seating position affects the stability.

● ● ●

## Transfers

See Figure C

### Over a threshold:

**C1.** Chair only, push down on the wheel attachment.

**C2.** With occupant, pull the chair backwards.

### In stationary position:

**C3.** Always lock as many castors as possible when the chair is to be used in a stationary position.

**C4.** Out-turned castors improve the chair's stability.

### Transferring:

Assess the risks and take notes. You as a carer are responsible for the safety of the user. Aids should be used if the transfer is deemed as risky. Visit www.etac.com for manual transfer aids.

**C3-C4.** It is important to ensure that the castors are locked and pointing outward before carrying out transfers to and from Clean.

- Remove the armrest on the transfer side of the chair.

**C5-C7.** Carry out the transfer.

● ● ●

English		Clean	Clean 24"	Clean HRB				
<b>Armrest</b> <span style="float: right;">See Figure D</span>		<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>				
<p><b>D1.</b> The armrests are swing away and detachable.</p> <p><b>D2.</b> The armrests are lockable.</p>								
<b>Height adjustment</b> <span style="float: right;">See Figure E</span>				<input checked="" type="checkbox"/>				
<p><b>E1-3.</b> The height can be adjusted with ease by placing the screw in the required position.</p> <div style="display: flex; justify-content: space-between;"> <div style="border: 1px solid black; padding: 5px; width: 40%;">  Check that all four castors are similarly adjusted, otherwise stability will be reduced.                 </div> <div style="border: 1px solid black; padding: 5px; width: 40%;">  Do not use the chair if the wing nuts are loose.                 </div> </div>								
<b>Footrest</b> <span style="float: right;">See Figure F</span>								
<p><b>F1.</b> The carer can easily push in the footrest with their foot.</p> <p><b>F2.</b> The footrest is detachable.</p> <p><b>F3.</b> The footrest has 4 different angle and height settings.</p> <p><b>F4.</b> The footrest has 8 different angle and height settings.</p> <p><b>F5.</b> More adjustment options are available by moving the pin.</p> <div style="display: flex; justify-content: space-between;"> <div style="border: 1px solid black; padding: 5px; width: 40%;">  Remove the pins once the footrest has been removed.                 </div> <div style="border: 1px solid black; padding: 5px; width: 40%;">  When the chair is adjusted to a height of 47.5 cm, the lowest footrest position must not be used.                 </div> </div>		<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>				
<b>Backrest upholstery</b> <span style="float: right;">See Figure G</span>		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>				
<b>User brake</b> <span style="float: right;">See Figure H</span>		<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>				
<b>Accessories</b> <span style="float: right;">See Figure I</span>								
<p>The accessories section shows all the options that are available for your shower chair. Variations from standard can occur in different markets. Limitations in combinations of accessories may occur. For more information visit <a href="http://www.etac.com">www.etac.com</a>.</p>								
See Figure I		Clean	Clean 24"	Clean HRB	See Figure I	Clean	Clean 24"	Clean HRB
<b>I1.</b> Pan	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<b>I15.</b> Soft back	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<b>I2.</b> Pan and lid with handle	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<b>I16.</b> Trunk support	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<b>I3.</b> Lid with handle	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<b>I17.</b> Heightening spacers	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<b>I4.</b> Disposable pan	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<b>I18.</b> Two 75 mm lockable castors	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<b>I5.</b> Pan holder	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<b>I19.</b> Rear wheel adaptor	<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>
<b>I6.</b> Bucket holder	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<b>I20.</b> Soap dish	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<b>I7.</b> Disposable pan holder	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<b>I21.</b> Crossbar	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<b>I8.</b> Attachment for low mounting of pan/bucket	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<b>I22.</b> Legrest, angle adjustable	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<b>I9.</b> Soft seat	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<b>I23.</b> Foot support, foldable and detachable	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<b>I10.</b> Comfort seat	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<b>I24.</b> Backrest upholstery	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<b>I11.</b> Comfort cushion	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<b>I25.</b> Safety belt	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<b>I12.</b> ROHO protect cushion, pressure-relieving	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<b>I26.</b> Positioning belt	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<b>I13.</b> Comfort padding	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<b>I27.</b> Chest harness belt	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<b>I14.</b> Armrest cushion, pair	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>				

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Etac A/S
R82 UK Limited.
R82, Inc.
Etac Supply Center AB