

# USER INSTRUCTIONS

## NUVO EXTRA WIDE TOILET FRAME

L70294

NUVO Extra Wide Toilet Frame

### ⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

### Maximum User Weight



## 1. INTENDED USE

To assist those who have difficulty in lowering onto, or rising from a standard toilet. They can also be used in conjunction with a raised toilet seat. An optional floor fixing kit is also available (L70324).



## 2. SAFETY

**WARNING:** The maximum user weight must not be more than 254 kg (40 st).

**DO NOT:** Attempt to stand or climb on the product.

**MAKE SURE:** All legs are set at equal height to avoid rocking.

## 3. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

### 3.1 Contents:

1. Frame Assembly x 1
2. Leg Extensions with Ferrules x 4
3. E-clips x 4

## 4. ASSEMBLY

This product is supplied fully assembled.

## 5. INSTALLATION

Adjust the height for the toilet and user before installation.

### 5.1 Adjust the Height

1. Remove the 4 x leg extension E-clips.
2. Slide the leg extensions to the required height, align the holes and replace the E-clips.
3. Place the toilet frame around the toilet.

**Note:** Make sure the seat and toilet frame are fully secure before use.

### 5.2 Fitting the Optional Floor Fixing Kit L70324

Please refer to the user instructions provided with the optional floor fixing kit.

## 6. MAINTENANCE

Check that all fixing clips are inserted correctly and that there is no excessive movement.

If a floor fixing kit has been fitted, check the fixing screws are secure.

Inspect the product regularly for signs of wear or damage, pay particular attention to the condition of the ferrules.

Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

## 7. CLEANING INSTRUCTIONS

Please check the suitability of the cleaning products before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.

Regularly clean with a non-abrasive cleaner.

To disinfect, wipe with a dilute bleach solution, and rinse thoroughly with fresh water.

This product is suitable for thermal decontamination and can be steam cleaned or autoclaved on cleaning cycles up to 70°C.

Allow product to fully dry prior to use/storage.

8. ACCESSORIES & SPARE PARTS

L70324 - Floor Fixing Kit

For further information please visit our website or call Customer Services on the details below.

9. PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from metal & plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

10. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

<b>NRS HEALTHCARE</b> Sherwood House, Cartwright Way, Forest Business Park Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.	
Tel: Customer Services: +44 (0)345 121 8111	Email: <a href="mailto:customerservice@nrshealthcare.co.uk">customerservice@nrshealthcare.co.uk</a>
<b>NRS Healthcare</b> is a leading supplier of community equipment and daily living aids. Please visit one of our websites to view our full range of products: <a href="http://www.healthcarepro.co.uk">www.healthcarepro.co.uk</a> or <a href="http://www.completecareshop.co.uk">www.completecareshop.co.uk</a>	