

User Instruction for BA-7252 (Soft Raised Toilet Seat-2” Height)

Please read and retain this instruction, it provides essential information for correct and safe use.

This instruction is guideline only.

INTRODUCTION

This Soft Raised Toilet Seat is used to raise the height of a toilet to a comfortable level, making it easier to sit down and get up. It fits most standard toilet seats and can raise their height by 2 inches. It can be held on the existing toilet seat by Velcro straps and is easily removed for cleaning.

SPECIFICATION

Overall Size	Height	Weight	Material
400 x 375 x 50 mm	50 mm	343 g	PVC / PU Foam

INSTRUCTIONS FOR USE

1. Prior to installation, ensure that the existing toilet seat is fitted securely and undamaged. Lift the lid of the toilet.
2. Place the soft raised toilet seat centrally on the top of the toilet seat and ensuring that the Velcro straps are tight with the toilet seat.

WARNING

- This Soft Raised Toilet Seat is not intended for permanent installation. The unit should be loosened and retightened once a week to ensure a proper fit.
- Supervision or assistance when using the raised toilet seat is recommended for users with limited physical capabilities.
- Do not hold onto the seat as a support when getting on and off the toilet seat.
- DO NOT use UV light to disinfect the product.
- Periodically inspect for wear and damage.

MAINTENANCE

Simply hand wash by cold or warm water with neutral detergent. Wipe with a dry cloth after washing.

