

**Maximum
User Weight:**



General safety Warning

Please ensure these instructions are fully read and implemented. Failure to do so may result in injury. Retain these instructions in a safe place for future reference.

1.0 Intended Use

The Height Adjustable Extra Wide Commode is designed to assist those who have difficulty in accessing or using a standard toilet. It can be used either as a commode or over the toilet. The extra width and height adjustment, accommodates a wider range of users than a standard width commode.

2.0 Safety Precautions

- Check all fixings are installed correctly.
- Ensure all adjustable legs are set to the same height and the spring clips securing the legs are fully engaged
- Ensure the product is always placed on a sturdy, level surface.
- Do not exceed the max user weight limit.
- Inspect at regular intervals, for any excessive wear or damage, if you see any do not use the product.

3.0 Unpacking

Remove packaging and check product carefully before assembling. Do not assemble if any parts are damaged.

4.0 Assembly & Use

- Secure the backrest to the commode frame using the fixings provided



- Adjust the seat to the correct height by depressing the spring button clips on the leg and sliding to the desired height.
- Re-engage the clips, ensuring all legs are set to the same height.
- Check the product is secure & stable before use.

Potty fitting and removal

Slide the potty in position under the seat, from the rear, making sure it is located on both rails underneath the seat.

To remove, slide the potty out and fit the lid provided to avoid splashing.

Carefully carry the potty and dispose of the contents into a proper sluice or WC.

Never use an open drain, sink or basin. This could cause a serious health risk.

Clean out any residue with a suitable disinfectant and refit to the commode after use.

5.0 Cleaning - General

Clean regularly using a general purpose, neutral detergent and warm water. Please check the suitability of the cleaning product before use. Do not use abrasive cleaners or cloths as this will permanently damage surfaces and invalidate any warranty. To disinfect, wipe with a dilute bleach solution and rinse thoroughly with fresh water. Allow the product to dry thoroughly prior to use/storage.

6.0 Maintenance

Check that all fixing clips are correctly located and there is no excessive movement. Inspect product regularly for signs of wear or damage, pay particular attention to the condition of the ferrules. Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

7.0 Product Disposal

Product should be thoroughly cleaned prior to disposal. Product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

8.0 Product Guarantee

NRS Healthcare, guarantee this product for a period of 12 months from the date of purchase against faulty materials and workmanship. This guarantee does not cover; abuse, misuse, interference or tampering with any part of the product. Any guarantee issued by NRS Healthcare does not affect your statutory rights.

Customers Who Purchased This Product Also Purchased: L99351 Classic Bio Care Bags

NO Liquid, NO Odour



*The new Bio-degradable Care Bag is a commode potty liner containing a special sheet that absorbs and gels waste. After using, the liner is simply removed, sealed with draw string and disposed, either by incineration or as the care bag is biodegradable into normal household rubbish. By gelling and containing the waste, the Care Bag reduces spills and contamination, eliminating the onerous and time consuming need to wash the commode potty. **Supplied in pack of 20.***

NRS Healthcare

Sherwood House, Cartwright Way,
Forest Business park, Bardon Hill,
Coalville, Leicestershire, LE67 1UB, UK

Tel: Head Office: 0330 100 0098

Tel: Customer Services: 0345 121 8111

Email:

customerservices@nrshealthcare.co.uk

NRS Healthcare is a Leading Supplier of Community Equipment and Daily Living Aids, to view our full range, visit our website:

www.nrshealthcare.co.uk

UI/P11619/09/17/Iss1