

R82<sup>®</sup> 


penguin<sup>™</sup>

**English user manual**

G  
B

## **GB** CONTENTS

Safety .....	4
Warranty .....	4
Maintenance .....	4
Penguin .....	5
Product identification .....	6
Measurement.....	7
Producer .....	7
Technical data.....	7
Distributor .....	7

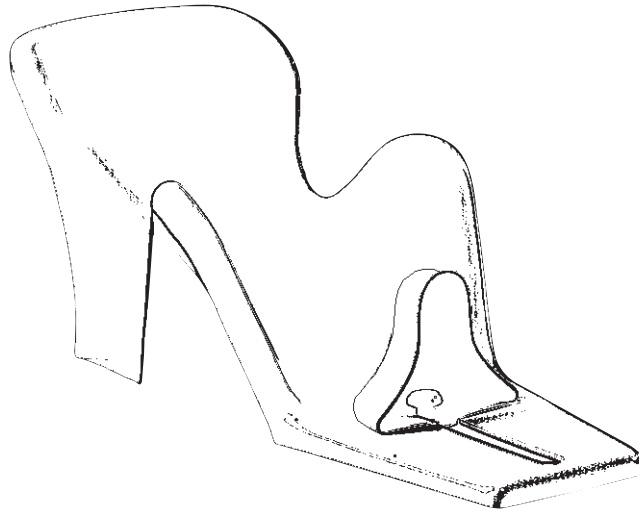


## **GB** PENGUIN

The Penguin lying seat from R82, can be used in the Orca bath tub as well as in a ordinary bath tub.

The Penguin lying seat is used when bathing and general nursing, but is also suitable when playing.

This manual will enable you to take full advantage of your Penguin.

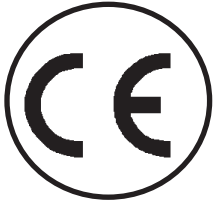

## **GB** SAFETY

The Penguin has earned the CE-mark. This certifies that it meets all relevant European safety requirements.

The durability of this product is 5 years when it is used on a daily basis. Hereafter the product must be renovated (by R82 personnel) to extend the lifetime.



Remove the CE-mark, when rebuilding the product or when using other than original R82 spare parts.



Never leave your child unattended in this product. Ensure constant supervision by an adult. Incorrect use of the product, may cause serious injury to the user. Ensure that the belt is securely fixed to the product before every use.

## **GB** WARRANTY

R82 offers a 2-year warranty against defects in workmanship and materials. The warranty would be adversely affected if the customer's responsibility of servicing and/or daily maintenance is not carried out according to the guidelines and intervals prescribed by the supplier and/or stated in the manual. For further information, we refer to the R82 homepage/download. The warranty can only be sustained if the R82 product is in use in the same country where it was purchased and if the product can be identified by the serial number. The warranty does not cover accidental damage, including damage caused by misuse or neglect. The warranty does not extend to non-durable parts, which are subject to normal wear and tear and need periodic replacement. The warranty is null and void if non-original R82 parts/accessories are used, or if the product is repaired or altered by anyone other than an authorised R82 representative or by trained personnel officially recognised by R82 for repair and maintenance of R82 products. R82 reserves the right to inspect the product being claimed for and the relevant documentation before agreeing to the warranty claim, and to decide upon whether to replace or repair the defective product. It is the customer's responsibility to return the item being claimed for under warranty to the address of purchase. The warranty is given by R82 or, subsequently, an R82 dealer.

## **GB** MAINTENANCE

The smooth surfaces make the Penguin easy to clean. Clean the Penguin after every use.



Do not use any cleaning material which contain chlorine.




Adjustments which are not described in this manual, must only be made by authorized R82 personnel.


## **GB** PENGUIN

- A) Arms and legs are easily moved when using the Penguin.
- B) The pommel is easy to adjust for the individual user.
- C) Suction discs are mounted, which make the Penguin stable on a smooth surface.



It is very important to keep watchful eye on the child when it is using the Penguin. Check the temperature of the water before placing the child when Penguin is used in a bath tub.

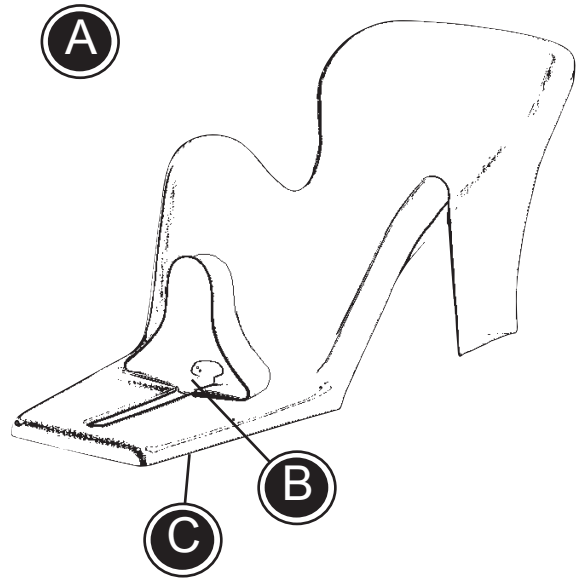

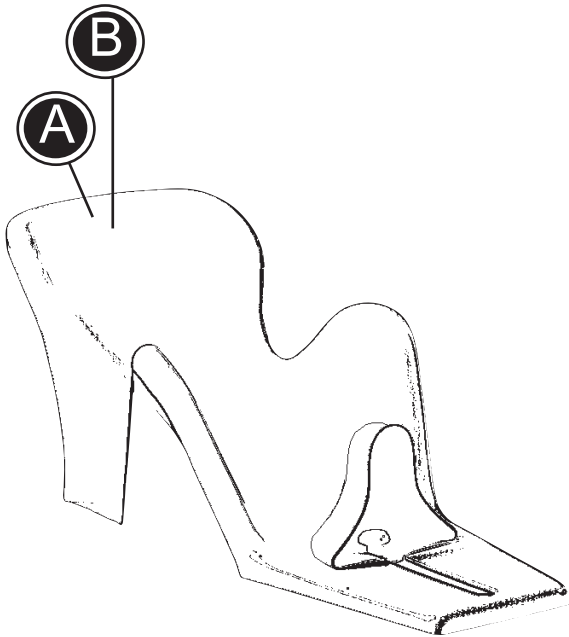

## **GB** PRODUCT IDENTIFICATION

### A) Serial number

The label is placed on the back of the chair.

### B) Manufacturer


The label is placed on the top of the back of the chair.

**A**



**B**



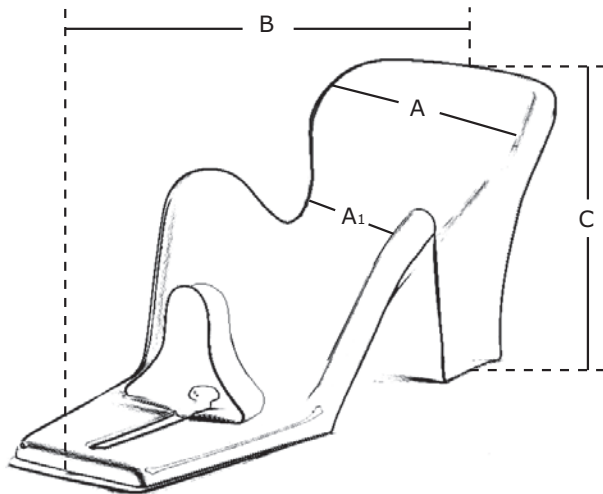

## GB MEASUREMENT

Width (A)	33 cm
Width, inside, smallest place (A1)	16 cm
Length (B)	80 cm
Height (C)	26 cm
Length, abdominal pad at top	56-72 cm
Weight	1½ kg

## GB TECHNICAL DATA

Plastic: PVC

## GB PRODUCER

R82 A/S  
Parallelvej 3  
8751 Gedved  
Denmark

## GB DISTRIBUTOR



Please find your distributor on [www.R82.com](http://www.R82.com)

