

Fitting instructions for Evacuation Sheet (EVS).

This Evacuation Sheet is designed to be permanently in place under a static anti-pressure mattress or an active mattress with a base layer.

- Remove the mattress from the bed
- Lay the Evacuation Sheet on the bed frame - straps side down.
- Replace the mattress on top of the Evacuation Sheet.
- Pull corner elastics over mattress to secure Evacuation sheet to mattress.
- The cross straps with buckles should be tucked into pockets provided.

In an emergency

- Remove cross straps with buckles at each side
- Clip buckles together across the patients chest and legs.
- Tighten to made the mattress like a canoe to secure the patient.
- With one carer at the foot end and one at the head end pull to safety by strong looped pulling straps

Thorpe Mill Ltd
Unit 1a Aire side Bus. Pk.
Royd Ings Ave, Keighley BD21 4BZ
Tel : 01535 682 630
Fax : 01535 682 639
Email : sales@thorpemill.co.uk