

G21808	Single 762mm (30") to 1220mm (48")
G21819	Double 1372mm (54") to 1525mm (60")

⚠ GENERAL SAFETY WARNING ⚠

Please make sure that you read and follow these instructions carefully. Failure to do so can result in injury. Keep these instructions in a safe place for future reference.

INTENDED USE

The Rise Easy Bed Aid is designed to assist the user in rising from, or sitting up in bed, without interfering with bed clothes. It is available in two sizes and is suitable for most beds including profiling beds. It can be used on either the left or right hand side of the bed.

SAFETY

Do not place obstructions (for example a bedside cabinet) adjacent to the bed rail. It is important that the arm is free to be moved away from the bed.

Do not use if feet of bed are too large to fit onto base plate pads.

It is recommended that this device is only used for clients up to 127Kg (20st).

MANUAL HANDLING GUIDANCE

During installation, it is necessary to lift the bed head onto the bed rail base plates. Please ensure that the person(s) installing the device is capable of doing this operation safely using the correct manual handling techniques and employ caution when lifting the bed.

ASSEMBLY & INSTALLATION INSTRUCTIONS

1. Remove all parts from the packaging. Check all parts are undamaged.
2. Ensuring there are no obstruction (E.g. Bed side cabinet) on the side which the client will exit the bed; move the bed approximately 300mm (12") from the wall
3. Place the 2 base plate sections on the floor and push both silver coloured connecting rods into both tubes on base plates to the bed size required.

Note: Do not pull past the warning dimples.

4. Slide assembly behind the head section of the bed and align base plates to bed castors/feet.

MAXIMUM USER WEIGHT



5. Lift the bed and slide assembly under the castors/feet of the bed. Lower bed onto the base plate ensuring the castors/feet sit onto the base plate. Carefully move the bed/Base plate assembly back to the wall.
6. The hand rail may now be fitted on the appropriate side of the bed. Slide the handrail over the location tube and push down until it hits the location stop pin.
7. Check that the bed rail feels secure and can be swung away from the bed.

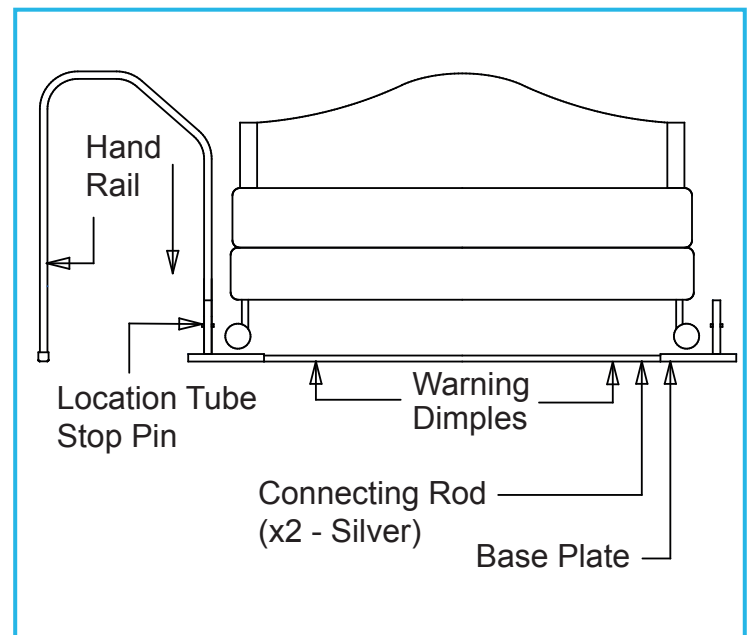


Diagram 1: Installation Diagram



Fig 1



Fig 2



Fig 3

USING THE RISE EASY BED AID

The Rise Easy Bed aid is designed such that the weight of the bed and client, applied on to the base plates, counterbalances the client during standing.

As it does not attach to the bed frame, it can be used on Divan beds (including sprung base), slatted beds and profiling/electrically adjustable beds provided the feet or castors can be fitted onto the base plate pads (within the sizes indicated; up to 5ft).

The normal “rest” position for the hand rail is shown in Fig 1, with the hand rail adjacent to the bed side. In this position, the bed rail may be used to assist the client to rise from lying to sitting and to aid repositioning in the bed e.g. turning. Please note: it is not intended to act as a cot-side to prevent the client from falling out of bed.

When standing from the bed, the hand rail can be left in the “rest” position against the bed, or swung out away from the bed as shown in Fig 2 and Fig 3, depending on the preferences of the client. When in the “swing away” position, the client should try to place their weight vertically down through the hand rail.

CLEANING

Clean regularly, wipe with a mild detergent/ disinfectant solution.

Do not use abrasive cleaners or cloths on this product.

PRODUCT DISPOSAL

This product should be thoroughly cleaned prior to disposal.

This product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 12 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover; abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS HEALTHCARE Sherwood House, Cartwright Way, Forest Business Park, Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.
Tel. Customer Services: 0345 121 8111
Email: customerservices@nrshealthcare.co.uk

NRS Healthcare is a Leading Supplier of Community Equipment and Daily Living Aids, to view our full range, visit our website:

www.nrshealthcare.co.uk

G21808-UI-4