

Product Technical Data Sheet 074 Slatted Bath Board Range

Product Range

M78342	Slatted Bath Board 660 (26")
M78354	Slatted Bath Board 686 (27")
M78366	Slatted Bath Board 711 (28")

Product Description & Intended Purpose

Designed to provide a safe and comfortable transfer platform for those who are unable to step in and out of the bath.



Product Features

- Reinforced PVC slats for durability.
- Tapered slats to encourage rapid drainage.
- Stainless steel fittings to prevent rust.
- Rubber pads for secure, non-damaging grip on bath.
- Optional grab handle for added support.



Technical Specification

Dimensions (mm)

- Length: model dependent.
- Width: 230
- Bracket to Board End: 25 — 90

Product Weight: M78389 2.22kg

M78391 2.37kg

M78408 2.5kg

Maximum User Weight: 30st (190kg)

Installation Information: *(Note: for detailed information refer to User Instructions)*

- Attach the brackets to the board by loosely screwing the hand wheels through the central slot, into the corresponding screw thread.
- Place the board across the top of the bath with the brackets facing up and the edge of the board pushed against the wall. Slide the far bracket into position, in line with the inside edge of the bath and tighten the hand wheel.
- Turn the board over with the furthest pads against the edge of the bath. Reach underneath and slide the nearest bracket pad up to the edge and secure using the hand wheel.

PTDS074

Slatted Bath Board Range

Safety Precautions

- Do not exceed the maximum user weight, 30st (190kg).
- Ensure the selected board is suitable for the bath, it should not protrude over the edge of the bath.
- Do not attempt to stand or climb on the bath board.
- It is recommended to position the board at the opposite end of the bath to the taps to allow plenty of room for safe transfers.

Cleaning

- Clean regularly using a general purpose, neutral detergent and warm water.
- To disinfect use a mild bleach solution and rinse thoroughly with fresh water.
- Please check the suitability of the cleaning product before use. Do not use abrasive cleaners or cloths as this will permanently damage the surface and invalidate any warranty.
- This bath board is suitable for thermal decontamination and can be steam cleaned or autoclaved on cleaning cycles up to 80°C.
- Always allow the product to fully dry prior to use or storage.

Care & Maintenance

- Check all fixings are secure and the board is tightly fitted to the bath.
- Check the condition of the rubber pads.
- Periodically check the frame for signs of wear. Any suspected faults or damage should immediately be reported to the supplier and the product taken out of action.

Accessories / Related Products

- M78378 Grab Handle — can be fitted to shower board for added support and user confidence.
- M78419 Slatted Bath Seat 150mm (6") — or alternative size, allows user to sit lower in the bath.

FAQ's

None

Product Supplied By:

NRS Healthcare Sherwood House, Cartwright Way, Forest Business Park, Bardon Hill, Coalville, Leicestershire. LE67 1UB UK