

User Instructions

Tuffet Leg Rest F21425

Maximum User Weight:

This product has a weight capacity of 51Kg (8st). As a guide, it is suitable for persons up to 153Kg (24st) to use as a leg rest when seated.

General Safety Warning

Please ensure these instructions are fully read and understood before installation/ use. Retain these instructions in a safe place for future reference.

Intended Use

The Tuffet Leg Rest is intended to support the legs of the user when sitting in a chair. The Tuffet can be used as a traditional leg rest (diagram1), or inverted for use as a "rocker" leg rest (diagram2).

Set up instructions

The Tuffet is fitted with a one way valve which allows the firmness of the footstool to be adjusted by inflating or deflating. To inflate, simply blow into the valve (as if blowing up a balloon). To deflate, insert a biro pen or similar object into the valve and the air will be released.

Cleaning Instructions

Wipe clean with a soft, damp cloth. Do not use an abrasive cleaner or cloth.

This product is from a range of over 3000 daily living aids available from Nottingham Rehab Supplies. To view the full range visit www.nrs-uk.co.uk or request a catalogue from the above number.

www.nrs-uk.co.uk



1- Traditional Leg Rest: In this position the Tuffet gives the user a choice of two elevations depending on which way round the Tuffet is placed



2 - Rocker Leg Rest: When the Tuffet is upturned it can pivot to the most appropriate position for the user



Manufactured in the UK for
Nottingham Rehab Supplies
Clinitron House, Excelsior Road, Ashby Park
Ashby de la Zouch, Leicestershire, LE65 1JG
United Kingdom

Tel: 0845 120 4522

Int'l: +44 (0) 1530 418378

Fax: 0845 121 8112

Email: customerservice@nrs-uk.co.uk