



User Instruction for HA-4128 (New Tablet Splitter)

Please read and retain this instruction, it provides essential information for correct and safe use.

This instruction is guideline only.

INTRODUCTION

The tablet splitter with blade cuts tablets in half for easy swallowing or for storage in small dose. Stainless steel blade neatly cuts tablets in half. It includes a compartment to store tablets.

SPECIFICATION

Overall Size	Weight	Material
83 x 44 x 23 mm	25 g	ABS / TPR(pad) / Stainless Steel(blade)

INSTRUCTIONS FOR USE

1. Open the top of the splitter.
2. Place the tablet in the middle of the V-shape notch.
3. Simply press down the top and the tablet will be cut in half.
4. Store the tablet which is not going to use immediately in the compartment in front of the V-shape notch.

WARNING

- Avoid direct contact your finger with the sharp blade.
- Capsule must be swallowed as a whole and never allowed to be cut.
- Keep out of the reach of children.
- Replace a new one if you find the blade become rusty.
- It would have a possibility to crash the pill when cutting some of relative soft structure pills
- DO NOT use in the dishwasher.
- DO NOT use UV light to disinfect the product.
- Periodically inspect for wear and damage.

MAINTENANCE

It should just be washed with normal cleanser and room temperature water.

