

User Instructions

P23038 NUVO Adult Toilet Frame

General Safety Warning

Please ensure these instructions are fully read and implemented. Failure to do so may result in injury. Retain these instructions in a safe place for future reference.

1.0 Intended Use

The NUVO Adult Toilet Frame is designed to assist users in lowering onto and rising from the toilet. It provides comfortable support for the user when getting on and off the toilet.

2.0 Safety Precautions

Do not exceed the maximum user weight of the product, 30st 190kg.

Do not attempt to climb on the product.

Not suitable for use by children.

3.0 Unpacking

Remove packaging and check all parts carefully before assembling the product. Do not assemble if any parts are damaged.

Contents:

- RH Frame
- LH Frame
- Crossbar X 2
- Leg Extensions X 4
- 'E-clips' X 8

4.0 Assembly

Attach the cross bars to the LH & RH frame;

- Push cross bar onto stub, align the holes and insert an 'e-clip' – Figure 3.

Ensure both cross bars are set to the same width.

Maximum User Weight

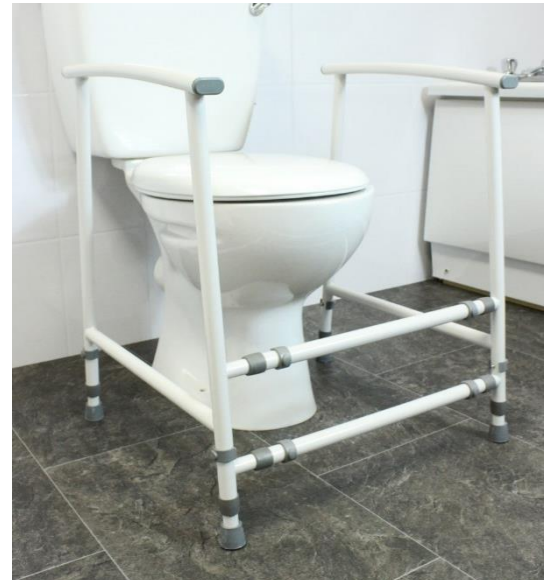


Figure 1 – Toilet Frame In Use



Figure 2 – Product Parts

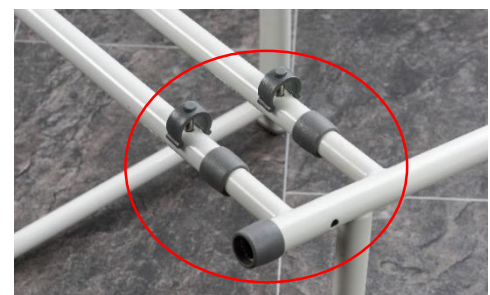


Figure 3 – Attaching the crossbars



Figure 4 – Positioning the leg extensions

To adjust the height:

- remove the 'e-clips' and slide the legs to the desired position, figure 4.
- align the holes and re-insert the 'e-clips', figure 5.

Use the markings on the legs to ensure that all four legs are positioned at the same height to prevent rocking and instability.

Place the frame around the toilet, ensuring it is pushed back as close to the toilet as possible, figure 1.

To adjust the width:

- remove the 'e-clips' and slide the cross bar to the desired position.
- align the holes and re-insert the 'e-clips'.

Ensure both cross bars are set to the same width.

5.0 Cleaning

Clean regularly using a general purpose, neutral detergent and warm water.

Please check the suitability of the cleaning products before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.

To disinfect, wipe with a dilute bleach solution and rinse thoroughly with fresh water.

Allow the product to dry thoroughly prior to use/storage.



Figure 5 – Attaching the leg extensions

6.0 Maintenance

Check that all fixing clips are inserted correctly and there is no excessive movement.

Inspect product regularly for signs of wear or damage, pay particular attention to the condition of the ferrules. Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

7.0 Spare Parts

P11668 – NUVO Plastic Fittings

For more details, please visit our website www.nrshealthcare.co.uk or contact customer services 0345 121 8111.

8.0 Product Disposal

Product should be thoroughly cleaned prior to disposal.

Product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

9.0 Product Guarantee

NRS Healthcare, guarantee this product for a period of 12 months from the date of purchase against faulty materials and workmanship. This guarantee does not cover; abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS Healthcare

Sherwood House, Cartwright Way,
Forest Business park, Bardonia Hill,
Coalville, Leicestershire, LE67 1UB, UK

Tel. Head Office: 0330 100 0098

Tel. Customer Services: 0345 121 8111

Email: customerservices@nrshealthcare.co.uk

NRS Healthcare is a Leading Supplier of Community Equipment and Daily Living Aids, to view our full range, visit our website:

www.nrshealthcare.co.uk

UI/P23038/05/18/Iss1