

## User Instructions

### Standard Bed Stick Range

#### M91115 – Standard Bed Stick – Chrome Plated Steel

#### M54234 – Standard Bed Stick – White, Powder Coated Steel



#### General Safety Warning

Please ensure these instructions are fully read and implemented. Failure to do so may result in injury to the user. Retain in a safe place for future reference.

#### Intended Use


The NRS Standard Bed Stick provides a stable support to help clients move themselves to a sitting position and assist them getting in and out of bed. This device is suitable for both divan and slatted domestic beds and is suitable for all bed sizes from single to king-size.

#### Unpacking

- This product requires assembly before use.
- No tools are required.
- Remove the product from its packaging
- Check for damage. If you see any, or suspect a fault, do not assemble, contact your supplier immediately.
- Check all parts are present - see contents list.

#### Contents

Bed Stick Long	1
Bed Stick Short	1
E-Clip	2

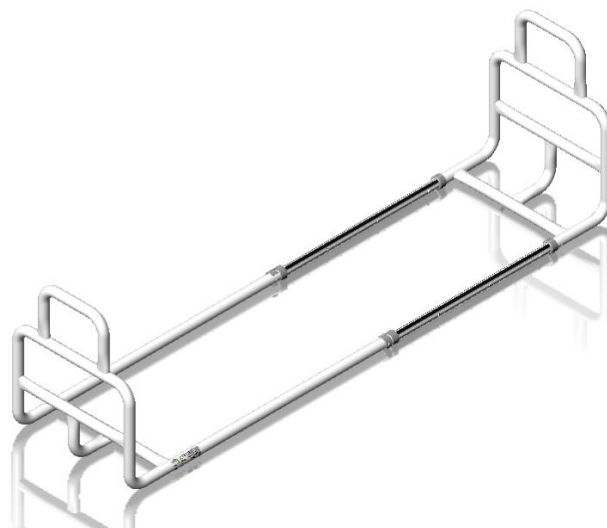

#### Safety Precautions

- This product is not designed for use with profiling/adjustable beds.
- Please ensure the user does not exceed the maximum user weight: 190kg (30 stones).
- Ensure the mattress is resting firmly against the bed stick handle.
- Check the bed stick is in the correct position and is stable and secure before use.
- **Important safety note:**

The user (from a lying position) should be able to reach the bed stick handle without stretching, but the handle must never be placed in a position too close to the headboard to cause a risk of head entrapment.

The position of the Bed stick should be based on a clinical assessment.

## Assembly Instructions

Assemble the Bed Stick by sliding the two chrome extension Tubes into the Long Bed Stick handle select the correct holes to suit the bed size, and secure with e-clips.

Place the bed stick onto the bed base, position the device (which should ideally be based on a clinical assessment) such that when placed under the mattress with the handle fitted, the client (from a lying position) should be able to reach the handle without stretching. Please also refer to the important safety note for positioning instructions.

Ensure the mattress is resting firmly against the Bed Stick Handles, to prevent the risk of entrapment.

Note the Handles can be reversed if not required on the opposite side of the bed, see diagram below (divan beds only.)


## Product Maintenance & Cleaning Instructions

- Clean regularly, wiping down with a soft cloth and a mild detergent/disinfectant solution
- Do not use abrasive cleaners
- This product can be autoclaved up to 80°C
- Inspect on a regular basis the stability and rigidity of the product.
- Check clips are fitted correctly and the product is positioned correctly on the bed.

## Product Guarantee

NRS Healthcare, guarantee this product for a period of 12 months from the date of purchase against faulty materials and workmanship. This guarantee does not cover, abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

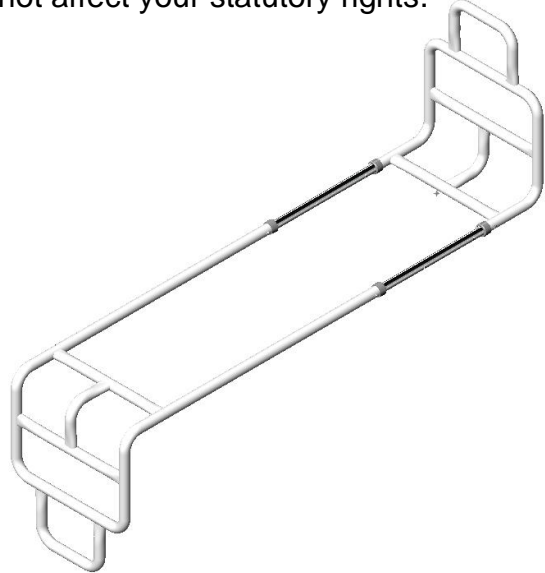

The **NRS Standard Bed Stick Range** is part of a range of over 3000 daily living aids available from NRS Healthcare.

To view the full range, please visit our website: **[www.nrshealthcare.co.uk](http://www.nrshealthcare.co.uk)** or request a catalogue from the number below.

Additional copies of this user instruction are available on our website: **[www.nrshealthcare.co.uk](http://www.nrshealthcare.co.uk)**

### **NRS Healthcare**

Sherwood House, Cartwright Way,  
Forest Business Park, Bardonia Hill,  
Coalville, Leicestershire,  
LE67 1UB,  
United Kingdom.

Tel. Head Office: 0330 100 0098

Tel. Customer Services: 0345 121 8111

Email: [customerservice@nrshealthcare.co.uk](mailto:customerservice@nrshealthcare.co.uk)

**NRS Healthcare** is a Leading Supplier of Community Equipment and Daily Living Aids, to view our full range, visit our website:

**[www.nrshealthcare.co.uk](http://www.nrshealthcare.co.uk)**