

USER INSTRUCTIONS

SPA GRAB RAIL RANGE

P01304	350 mm (14") SPA Stainless Steel Straight Grab Rail	P01339	350 mm (14") SPA Stainless Steel Curved Grab Rail
P01316	480 mm (19") SPA Stainless Steel Straight Grab Rail	P01341	480 mm (19") SPA Stainless Steel Curved Grab Rail
P01328	620 mm (24.5") SPA Stainless Steel Straight Grab Rail	P01353	620 mm (24.5") SPA Stainless Steel Curved Grab Rail

⚠ General Safety Warning ⚠

Read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

1. INTENDED USE

The SPA Grab Rail range provides a stable support to assist users to raise and lower themselves, and provides stability when overcoming everyday obstacles.



Figure 1: Curved Rails



Figure 2: Straight Rails

Safe Working Load



2. SAFETY

⚠ Warning: Do not exceed the Safe Working Load.

- The person that installs this product must be suitably qualified and competent.
- Use only as a support to assist movement. Do not apply full weight to the rail.
- Never drill through pre-stressed concrete to install a support rail.
- Always check for concealed electrical cables and water pipes when installing.
- Check your legal requirements relating to earth bonding of metal parts.
- Inspect product (to ensure the rail is unbroken, rigid, stable, and that the fixings & screws are fitted into place) at regular intervals. See section 5.2.

3. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

3.1 Contents:

1. Grab Rail
2. Cover Plate (pre-fitted)
3. Wall Bracket (pre-fitted)

Note: Fixings are not supplied.

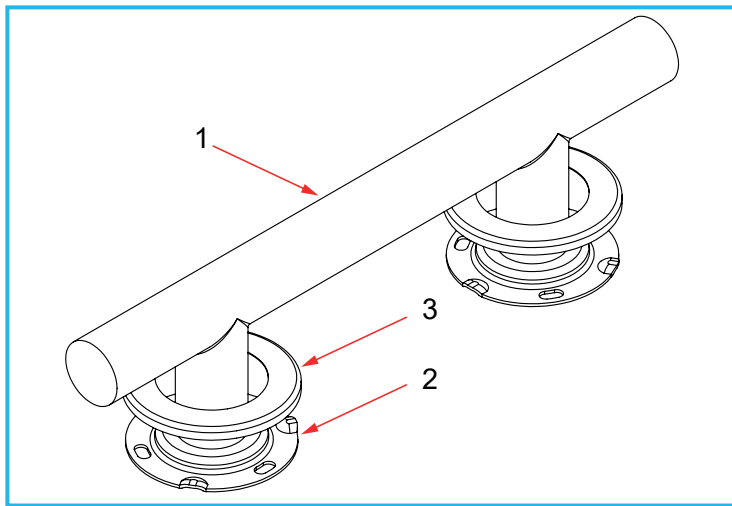


Figure 3: Parts

4. ASSEMBLY/INSTALLATION

4.1 General

- Install the rail in an area where the mounting surface and substrate are structurally sound.
- Check the type of substrate you intend to fix the rail to. For example; brick, timber or concrete.
- Only use fixings suitable for the substrate.
- **Note:** As we do not know the substrate, we cannot recommend any particular fixings. If in any doubt, ask a reputable fixings supplier for advice.

4.2 Installation

1. Hold the rail against the fixing surface and mark the hole positions.
2. Drill holes suitable for the fixings and screws you are to use.
3. Hold the rail against the wall and secure using your chosen fixings.



Test the installation and make sure the rail is rigid, stable and fit for purpose.

4. Conceal the fixings by pushing the cover plates over the wall brackets.

5. MAINTENANCE

Regularly check the product is securely fixed to the wall and fit for purpose.

Check all fixings are tightened and in good condition.

If you see damage, or suspect a fault, do not use. Contact your supplier.

6. CLEANING INSTRUCTIONS

Check the suitability of the cleaning products before use.

Clean regularly. Wipe down with a soft cloth and a mild detergent/disinfectant solution.

DO NOT use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.

7. PRODUCT DISPOSAL

This product is manufactured from stainless steel can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

8. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

9. GLOSSARY

	Maximum User Weight		Warning
--	---------------------	--	---------

NRS HEALTHCARE Sherwood House, Cartwright Way, Forest Business Park
Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services:+44 (0)345 121 8111

Email: customerservice@nrshealthcare.co.uk

NRS Healthcare is a leading supplier of community equipment and daily living aids.
Please visit one of our websites to view our full range of products:
www.healthcarepro.co.uk or www.completcareshop.co.uk