



installation

1. Place tray on flat surface with bottom side of tray facing upward.
2. Insert top tube of mounting frame A (Fig 1) into top tube of mounting frame B.
3. Push frames together to its narrowest position.
4. Insert flat plates of mounting frame into groove (Fig 2) in the center of tray.
5. Loosen nuts (Fig 3 A).
6. Slide grooved end of mounting brackets over screws.
7. Snap front plastic clips (Fig 3 B) onto Trotter frame.
8. Tighten nuts.
9. Adjust mounting frame width to align with mounting bracket tubes (Fig 3 C).
10. Tighten screw (Fig 2 A).
11. Insert mounting frame into mounting bracket tubes.
12. Tighten knobs (Fig 3 D).

adjustments

1. Adjust depth by loosening knobs (Fig 3 D) and pushing or pulling tray to desired position.
2. Tighten knobs (Fig 3 D)

